



Re-Order Number	Description	Size	Units
703082	Solid Seat Platform	18"x 16"x 1"	Each

### **Purpose**

Several models of wheelchairs utilize a seat foundation consisting of a fabric or vinyl that is attached on both sides of the wheelchair with a screw assembly. This fabric covering drapes or hammocks in time and this net effect prevents the patient from maintaining correct posture unless a solid seat insert or cushion with a rigid bottom is placed on the hammocked seat. This solid seat insert is intended to be placed over the hammocked seat and under the cushion. It is not intended to be used on a solid seat wheelchair or under a solid bottomed cushion. This product is not intended to be used without a wheelchair cushion.

### **Easy to Use**

Place on top of the hammocked seat and under the cushion. Do not have the resident sit directly on this solid insert. It is to be centered on the seat and pushed as far as possible to the back of the wheelchair. If space allows, it can be placed inside the cushion cover below the cushion.

### **Easy Care**

The cover is heat sealed vinyl and can be cleaned with a spray and wipe cleaner. Do not utilize harsh cleaners as they will adversely affect the vinyl.