

# Hydrocover Storage Rack

## Item #081505312

Thank you for purchasing the Sammons Preston Hydrocover Storage Rack.

The parts included are:

(1) Powder-Coated Base (6) Stainless Hooks (12) Vinyl Caps and (6) Stainless Screws

The Hydrocover Storage Rack is designed for mounting to 16" centered wood or metal studs. In order to mount the rack to the wall, choose the proper location and locate the wall studs. Utilize the base as a template and mark the wall. Drill a small pilot hole for each of the screws for easy attachment. Make sure the hooks are facing the floor, and then secure the base in place with the screws provided. If your wall is not able to have both studs on 16" centers, then you will need to use different wall mounting hardware to fit your requirement.

As soon as the base is secure, slip each hook through the upper and lower holes of the base, and then press the vinyl caps onto the bottom of the hooks.



**SAMMONS  
PRESTON**  
Sammons Preston  
A Patterson Medical Company  
1000 Remington Blvd., Suite 210  
Bolingbrook, IL 60440  
Phone: 800-323-5547  
Direct Phone: 630-378-6000  
Fax: 800-547-4333  
Direct Fax: 630-378-6310



**SAMMONS  
PRESTON**  
Sammons Preston  
A Patterson Medical Company  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012



Authorized European Representative  
**Homecraft Rolyan**  
A Patterson Medical Company  
Nunn Brook Road, Huthwaite  
Sutton-In-Ashfield, Nottinghamshire NG17  
2HU  
United Kingdom  
Phone: 08702 423 305  
Fax: 08701 660 211

**International**  
Phone: +44 1623 759 146  
Fax: +44 1623 726 886

© April 2009  
Patterson Medical Products, Inc.