EXTRA WIDE

STEEL

Rollator

WITH LOOP LOCK BRAKES

Getting Around
Has Never
Been Easier...

 **DMI** DURO-MED INDUSTRIES®

INSTRUCTIONS FOR ASSEMBLY

WARNING! It is very important that you read all instructions before assembly and/or use of this Rollator.


Before using the rollator, be sure your weight is within the weight capacity of the unit. This rollator will require little or no assembly and can be performed without tools.

Parts of Assembly:

2 Handlebars with Loop Lock Brakes; 1 Basket; 1 Backrest

ASSEMBLY INSTRUCTIONS:

1. Open your rollator. Push the folding hinges down until they lock securely in place on both sides. They are located under the folding seat.
2. Place both loose hand pieces, with the pre-assembled locking loop brakes, into the handlebar openings and adjust to the desired height. To do this, remove the knob, washer and screw from the handlebar and insert the screw, with the washer on it, through the desired hole aligned in the rollator bar and screw the knob back on securely.
3. Insert removable backrest into rollator frame securely by making sure the retainer pins are locked into position through the frame holes on both sides.
4. Test and adjust the tension on the braking system mechanism to ensure proper and secure braking before use.
5. If your rollator comes with a basket, secure the basket in the proper location. Flip up seat and place basket on rollator crossbars.

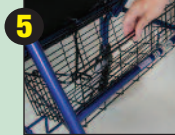ITEM # 501-1032 SERIES

SPECIFICATIONS

Basket: 20" x 8" x 6" deep

Wheels: 8"

Overall Width: 30.5"

Width Between Handles: 22"

Handle Height: 35" - 39"

Seat: 18" x 13.25"

Seat Height: 22"

Weight: 28 lbs.

Weight Capacity: 375 lbs.

USE OF NEW ROLLATOR

**BEFORE USE, PLEASE READ ALL CAUTION
AND WARNING LABELS ON THE ROLLATOR**

WALKING

This unit is to be used as a mobility aid creating a natural balance. With hands grasped securely onto handles, the user walks slowly behind the rollator. Gently, push forward with weight distributed evenly between handlebars.

BRAKING

Normal Brake can be activated by pulling the brake handle up, similar to operating the brakes of a bicycle. This function may be applied, while walking with the rollator to slow down momentum, stop for a moment, or maneuver up, down or around boundaries which may be in your path.

Parking Brake can be activated by pushing the handle down into the lock position. This function should be activated when the user stops for a period of time. For safety reasons, the parking brake should always be engaged before sitting down onto the seat. Also if shopping, it will be a good idea to activate the parking brake while stopped in an aisle, to avoid rollator from advancing forward. To release the parking brake, gently pull up.

The braking system is convenient and easy to operate. People who may have hand disabilities should find the brake handles wide, comfortable and non binding. Minimum pressure will need to be applied to activate the locking loop brake system.

FOLDING

Baskets should be removed before folding any rollator. From the user side of the unit grasp the folding strap, located under the hinged seat, and lift into the folded position. The rollator will fold easily for storage or transport.

MAINTENANCE

Always make sure your brakes are operating correctly before use. Keep the brake handles free from dirt, dust and grime. Build-up of particles could cause your brakes not to engage properly. If the brakes are not braking, check the cables for looseness. The cables can be tightened at the adjuster screw located where the cable is connected to the brake handle. If the cable is broken, it will need to be replaced. Periodically check and tighten all nuts, bolts, screws, and other hardware located on rollator.

WALKING MADE EASY

SQUEEZE TO STOP

RELEASE TO ROLL

PUSH DOWN TO LOCK

LIFT SEAT AND PULL
STRAP TO ENGAGE
FOLDING MECHANISM

LIMITATIONS AND EXCLUSIONS

Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond reasonable control are not covered by this warranty.

Parts or materials which are subject to normal wear resulting from the use of this product, and which must be replaced or repaired due to such normal wear, are the owner's responsibility and are not covered by this warranty.

This warranty is exclusive and is in lieu of all other express warranties, implied warranty, including but not limited to the implied warranty of merchantability and fitness for a particular purpose, and shall not extend beyond the duration of the express warranty provided herein. DMI will not be liable for consequential or incidental damages whatsoever.

This warranty gives you specific legal rights which may vary from state to state. Some states do not allow the exclusions or limitations of incidental or consequential damages, or how long an implied warranty lasts, so the exclusions and limitations may not apply to you.

WARNING! READ BEFORE USING

This unit is not intended to be used as a wheelchair to transport user. At regular intervals check and tighten all fasteners and hinged parts. Check all plastic parts for wear often. Units are not designed to assist users from a sitting to a standing position. When stopping, do not lean excessively on the unit. The brakes are not designed to withstand excessive pressure. Excessive use causes the brake cables to stretch or fray and adjustment or replacement of the brake cable may be required. DMI rollators are not suggested for people with severely limited ambulation for whom a rigid walker may be recommended. Please consult with your physician before using this product especially if you are taking any medication. Failure to observe this warning may result in injury.

CAUTION! CHECK BRAKES BEFORE USING

Always check your brakes before using your rollator. Keep your brake handles in excellent condition at all times. Always check for loose parts, tightness of the cables and cleanliness. It is best to keep your rollator fully assembled at all times.

Distributed by:
Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.mabisdmi.com

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©2009 Duro-Med Industries