




Six-Month Thin Bed Sensor Pad

Setup

This side up for optimal results.

Six-month unconditional warranty.

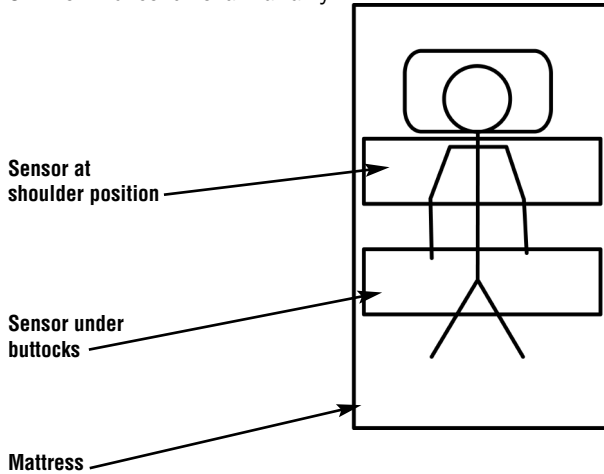

Setup instructions

1. Record initial use date here with non-erasable felt tip marker _____
2. Record expiration date here with non-erasable felt tip marker (six months from initial use date) _____
3. Remove paper backing from adhesive strips and place sensor pad on mattress as shown in the diagram above. For earliest detection, place sensor pad under patient's shoulders.
4. Route sensor cable to the alarm unit.
5. Connect sensor cable to alarm unit.

SERIAL # _____

THIS SIDE UP

Caution:

Do not fold or bend this sensor pad!

Before using this sensor pad, read entire instruction sheet and save for future reference. Failure to do so may result in injury or death to a person in your care.

Testing the system

1. Turn on the alarm unit. Ensure that the ON light indicates that it is active and the low-battery warning light is not blinking (if it is, change the battery).
2. Make sure the sensor cable is correctly inserted into the alarm unit.
3. Test pad by applying pressure to pad for three seconds, then releasing.

Note: Always test alarm and sensor pad together as a system to ensure operation.

Note: Visually inspect the sensor pad, cable, and alarm unit for any damage. Do not use if damaged; replace with undamaged unit.

Caution

1. This device is not a substitute for routine visual monitoring of patients. It will not prevent falls. Staff must make frequent routine inspections to ensure that the sensor pad remains correctly positioned under patient and that the alarm unit remains connected with adequate battery power.
2. Use of this pad may not be suitable for all high risk patients.
3. Sensor pad is intended to be used for individual patients for the time period indicated under the warranty only.
4. We recommend use of this product not exceed six months.
5. If damaged, replace—do not use.

Care and Maintenance

- Sensor pad is for single patient use only
- Do not bend, roll, fold, or immerse sensor pad in any solution. Store flat
- Clean using standard cleaning disinfectants. Do not heat-sterilize pads
- Do not tear or puncture vinyl pad cover
- Do not use equipment that has been damaged
- Dispose of all expired, soiled, damaged or contaminated sensor pads in accordance with established laws

Warranty

- Manufacturer warrants AliMed® Thin Bed Pressure Sensor Pad for six months from date of purchase for materials and workmanship. Warranty remains in force provided there is no evidence of misuse or improper installation
- Warranty is void if unit has been opened, damaged, or submerged

For further technical information on the use of this product call your supplier or

AliMed® at 1-800-225-2610 www.AliMed.com

- #71647 6-Month Thin Bed Sensor Pad & IQ Duo Alarm
- #70070 6-Month Thin Bed Sensor Pad & TR2 Alarm
- #79370 6-Month Thin Bed Sensor Pad & VOICE Alarm™
- #79366 6-Month Thin Bed Sensor Pad
- #79367 6-Month Thin Chair Chair Pad, 10/cs
- #77141 Sensor Basic Alarm
- #79990 TR2 Tamper-Resistant Alarm
- #2032 VOICE Alarm



www.AliMed.com

Made in Taiwan
#5905 (10.08)