Rev. 2 - 08/10_103640

Bauerfeind AG
 Weissendorfer Strasse 5
 07937 Zeulenroda-Triebes
 Germany
 Phone +49 (0) 36628-66-40 00
 Fax +49 (0) 36628-66-44 99
 E-mail info@bauerfeind.com

Motion is Life: www.bauerfeind.com

AchilloTrain

Aktivbandage
 Active support
 Orthèse active
 Vendaje activo

Motion is Life: www.bauerfeind.com

Material
Silikon*
Polyamid (PA)
Elastodien (ED)
Baumwolle (CO)
Viskose (CV)
Elastan (EL)
Polyurethan (PU)
Polyester (PES)
synthetischer Kleber
Druckfolie

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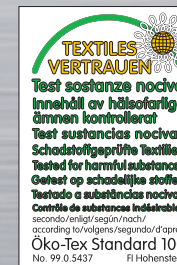
* Heel area – silicone consisting of casting compound, transparent, PK 69 A and resin PK 69 B
 Toe area – silicone consisting of casting compound, blue, PK 19 A and resin PK 19 B

Equilibrium pillow – silicone consisting of casting compound, transparent, P 7664 A and the resin P 7664 B were *not* accounted for in creating the material component description.

Enthält Naturkautschuklatex (Elastodien)
 Caution: This Product Contains Natural Rubber
 Latex Which May Cause Allergic Reactions.

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GB Dear client,

thank you for choosing a product from Bauerfeind. AchilloTrain is a high-quality medical product.

Please read through these instructions for use carefully. If you have any questions please contact your doctor, your local retailer or our Technical Service department.

AchilloTrain, the active Achilles tendon support is anatomically knitted which ensures an excellent fit. A viscoelastic profiled insert in the Achilles tendon area and the viscoelastic heel wedge provide relief for the Achilles tendon, soothing pain. The massage effect of the shaped insert promotes the circulation and helps reduce swelling and bruising.

Important: The supplementary heel pad must be worn on the healthy foot to compensate leg length.

⚠ AchilloTrain must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Painful irritation of the Achilles tendon (achilodynia) and adjacent tissues
- Follow-up management in Achilles tendon tears

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses² applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical retailer:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting AchilloTrain

1. Fold the upper part of the support outwards until the heel wedge can be gripped.
2. Pull the support over the foot until the heel is at the heel wedge.
3. Then pull up the folded-over part of the support taking care that the profiled insert is to the side of the Achilles tendon.
4. Adjust the fit of the support as necessary.
5. **Do not forget:** Insert the supplied compensation pad in the shoe of the healthy foot in such a way that it is flush with the heel area of the shoe.

⚠ Important Information

- AchilloTrain is a prescribed product that should be applied under a physician's guidance. AchilloTrain should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.

- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- The heel wedge can be removed from the product if necessary. To do this, turn the support inside out and extract the heel wedge through the opening in the velour pocket. If you do remove the heel wedge, do not use the offsetting heel cushion on the unaffected foot.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical advisor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.
- Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Washing instructions and product care

⚠ All Train products² can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing.

We recommend using a mesh bag (spin cycle: maximum of 500 revolutions per minute). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been tested by our in-house quality management system. However, should you have any complaint, please contact our local expert adviser or our customer service department.

Version: November 2009

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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