

Rolyan® Dorsal Night Splint

081607407 – SMALL/MEDIUM

081607415 – LARGE/EXTRA LARGE

Item Number	Size	Male Shoe Size	Female Shoe Size
0816-07-407	Small/Medium (left or right foot)	5 - 9	6 - 10
0816-07-415	Large/Extra Large (left or right foot)	9.5 - 14	10.5 - 15

Fitting:

- Unfasten straps. Slide splint over foot.
- Base of toes should be positioned near outer edge of foot pad.
- Adjust straps for comfortable and supportive fit.
- Straps will need to be re-adjusted regularly to provide support after patient movement.
- Do not over tighten straps.

Care:

Rolyan Dorsal Night Splint can be hand washed with mild detergent. Rinse and air dry only. Do not remove plastic shell from splint.



Patterson Medical
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310

Patterson Medical
National Service Center
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative
Patterson Medical, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

99104.711