

Homecraft Heavy Duty 3-in-1 Commode

0816-10-823



Before Installation

Remove the commode from package, making sure that it has not been damaged during handling and shipping. Contact Patterson Medical if you find damage.

Read the following instructions and make sure you understand them. For clarification of any step, consult a healthcare professional.

Installation

Attach the back and arms onto the frame. Place the Commode on a level floor. Determine the leg height that will ensure the safety and comfort of the user. To adjust: depress the spring button on the leg, slide it to the proper hole and release the button. While adjusting the other three legs, make sure the buttons are set at the same height as the first.

At this point and before each use, make sure that buttons protrude all the way through the holes so the height is locked. To double check the security, try to move the legs up and down; if the legs move, the buttons are not locked.

Set the commode on a level floor, where it will be stable and thus safe for the user. Any instability puts the user at risk of sliding off the commode and falling.

Application

Check the stability of the commode again. To maintain balance when sitting down or getting up, position both hands on the armrests. After being seated, push back on the seat until knees are against the frame.

The Commode may be used over a standard toilet for support and as a toilet riser. Remove the bucket and insert the splash guard. With the toilet lid up, place the Commode over the toilet. When correctly positioned, the splash guard should be in the toilet bowl.

Precautions

Supervision or assistance should be provided for users who are weak or tend to be unbalanced.

Before each use, ensure that spring buttons protrude all the way through the appropriate holes so they are locked in place.

Also confirm that legs are set securely on the floor and that they are set at the same height.

Test for stability before each use. Do not exceed the user weight capacity of 352 lbs.

Care

Clean the commode using basic cleaning/disinfecting products. Do not use abrasive, scouring pads or solvents including lacquer thinner, turpentine or acetone.

Periodically inspect the rubber tips on the legs. If tips are torn or missing, replace them immediately.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Patterson Medical

1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310

Patterson Medical

National Service Center
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative

Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom

Phone: 08444 124 330

Fax: 08448 730 100

International

Phone: +44 (0) 1623 448 706

Fax: +44 (0) 1623 448 784

© January 2013

Patterson Medical
Products, Inc.
081610823LIT