

KneeCrank II™

Directions for Application, Use and Care

Application

The low-profile KneeCrank II is an ideal way to apply and control force during the last few degrees of stretching a knee contracture. The accessory wedge may be used under the thigh to increase the degree of leg extension.

Use

1. Position the beveled end of the KneeCrank II under the buttock of the affected knee so that the heel of the foot is comfortably resting on blue foam.
2. The leg straps should be placed in a comfortable area above and below the knee. Apply pressure to the contracture by pulling the straps snug over leg and through the D-ring. Do not overtighten.
3. The straps may be marked with a pen to show progress.
4. Consult your physician for wearing time.

Care

Wipe clean with a damp cloth.

To Reorder

#62471 KneeCrank II
#62472 KneeCrank II, 2/cs

Distributed by:



AliMed® inc.
Dedham, MA 02026-9135

#2916/59
©1999 AliMed® inc. All rights reserved.

KneeCrank II™

Directions for Application, Use and Care

Application

The low-profile KneeCrank II is an ideal way to apply and control force during the last few degrees of stretching a knee contracture. The accessory wedge may be used under the thigh to increase the degree of leg extension.

Use

1. Position the beveled end of the KneeCrank II under the buttock of the affected knee so that the heel of the foot is comfortably resting on blue foam.
2. The leg straps should be placed in a comfortable area above and below the knee. Apply pressure to the contracture by pulling the straps snug over leg and through the D-ring. Do not overtighten.
3. The straps may be marked with a pen to show progress.
4. Consult your physician for wearing time.

Care

Wipe clean with a damp cloth.

To Reorder

#62471 KneeCrank II
#62472 KneeCrank II, 2/cs

Distributed by:



AliMed® inc.
Dedham, MA 02026-9135

#2916/59
©1999 AliMed® inc. All rights reserved.