

Supreme Sock Aid

Makes putting on socks easier for people who have difficulty bending

- Smooth inner surface allows the foot to glide effortlessly into the sock
- Outer surface has a slip resistant coating that holds the sock in place until the foot is fully inserted
- Three-finger design bends to accommodate the contours of the foot
- Loop strap enables one or two handed use; adjustable up to 30" (76 cm)
- All plastic design eliminates the wear and tear associated with cloth sock aids



Item 738450000

Made in Taiwan



Instructions:


1. Slide the sock over the end of the sock aid that does not have straps. (You may find it easier to do this if you bunch up the sock to the heel and then slip the sock aid into the sock.)
2. Straighten the sock and slide the opening closer to the top of the sock aid. The sock aid should be fully inserted into the sock as far up to the toe as possible.
3. While sitting down, hold the strap and place the aid on the floor in front of your feet.
4. Slip your toes into the sock and pull on the strap until your foot is completely inside.
5. Continue pulling until the sock is all the way on and the aid slides out of the sock.

Strap Assembly:

There are different slot markings on each side of the sock aid. Slots 1 and 3 are located on the side of the sock aid with the soft touch material. Slots 2 and 4 are located on the smooth side of the sock aid. (Please note: Slots 1 and 4 are the same slot.)

1. Take one end of the strap and slide it through Slot 1.
2. Once through Slot 1 slide down through Slot 2.
3. Once through Slot 2 slide up through Slot 3.
4. Slide end of strap down through Slot 4 to lock in place.
5. Repeat on the other side of sock aid with the other strap end.

Note: Strap length can be adjusted by initially inserting more or less of the strap through Slot 1.

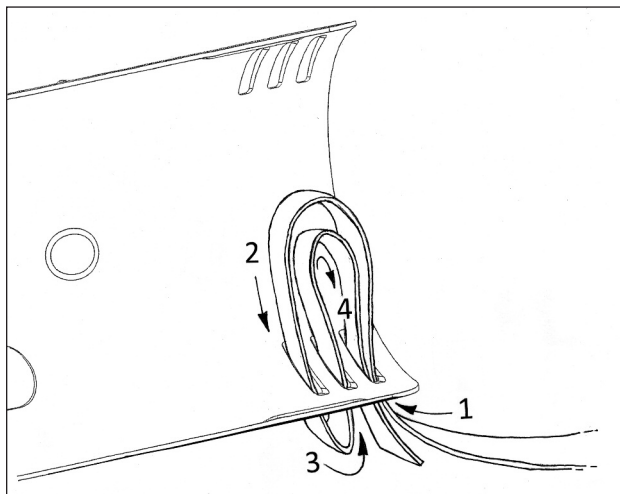

Cleaning:

Top rack dishwasher safe. Can also be cleaned with soap and water, household disinfectant sprays, etc.

Warranty: This product is guaranteed for 12 months for parts and labor under SP Ableware's standard warranty policy. For full details, and to register go to http://bit.ly/maddak_warranty.

