



Versa Form® Instruction & Care Information

Positioning Products that are Versatile and Reusable

Versa Form pillows are filled with small styrene beads which conform to the shape of the client. When air is extracted, using a vacuum pump, Versa Form molds to the individual body shape forming a firm support. When the valve is released, allowing air into the pillow, Versa Form regains its flexibility and is ready to be shaped again.

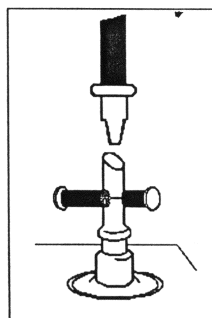

Step 1
Insert nozzle of the vacuum pump and begin to pump, molding the Versa Form pillow to the desired shape as air is extracted. You control the rate at which the pillow firms. Make sure the valve is open as illustrated.

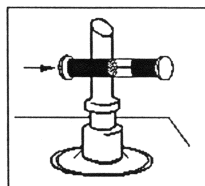

Step 2
When Versa Form reaches the desired shape, close the valve and remove the vacuum pump line, as shown.

To return Versa Form to its original state for reuse, open valve.

Versa Form:

- Holds its shape for days, if desired.
- Does not constrict circulation.
- Is available in several different sizes to accommodate you and your client's needs.
- Is durable Polyvinyl for repeated use.

Vacuum Pump 2825. The easy to use vacuum pump is attached to the valve on the Versa Form. Within seconds the air can be pumped out while molded to the shape you desire. When Versa Form reaches the desired shape, close the valve and remove the vacuum pump line.

Versa Form is available in five sizes (White):

- 2820** 22" x 12" (56 x 30.5 cm)
- 2821** 22" x 26" (56 x 66 cm)
- 2822** 22" x 34" (56 x 86 cm)
- 2823** 16" x 20" (41 x 51 cm)
- 2805** 32" x 40" (81 x 101.5 cm)
- 2820K** Versa Form Repair Kit (White)

Versa Form Plus Pillows retain their shape longer and have a longer life cycle. Versa Form Plus is available in five sizes (Blue):

- 2830** 22" x 12" (56 x 30.5 cm)
- 2831** 22" x 26" (56 x 66 cm)
- 2832** 22" x 34" (56 x 86 cm)
- 2833** 16" x 20" (41 x 51 cm)
- 2834** 32" x 40" (81 x 101.5 cm)
- 0815-26-086** Versa Form Plus Repair Kit (Blue)

Care of your Versa Form

- Inspect Versa Form before and after every use for possible damage.
- Do not lift by the valve.
- Close valve cap when wiping cushion with disinfectants.
- Do not repair Versa Form with adhesive tape.

Avoiding Leaks By using the covers for use and storage you can avoid leaks.

Testing for Leaks

- Apply suction to the cushion and close the valve.
- Examine 24 hours later. If it has completely lost firmness, a leak is suspected.

Locating Leaks

- Inflate the cushion until it becomes soft (do not over inflate).
- Submerge in sink and look for bubbles.

Repair

- A spare valve is available. Order **2820V**.

Warranty

- One year.

Cover can be used during use of product or for storage only. Versa Form is durable and will last 3-5 years when used several times a week.



Sammons Preston
A Patterson Medical Company
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European Representative
Homecraft Rolyan
A Patterson Medical Company
Nunn Brook Road, Huthwaite
Sutton-In-Ashfield, Nottinghamshire NG17
2HU
United Kingdom
Phone: 08702 423 305
Fax: 08701 660 211

International
Phone: +44 1623 759 146
Fax: +44 1623 726 886

© March 2010
Patterson Medical Products, Inc.