






Half Shifter, #92 7349

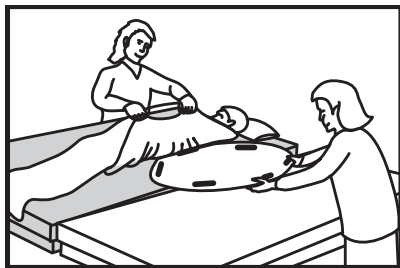

Lock stretcher against patient bed. Gently pull draw sheet up. Position the Half Shifter under draw sheet.

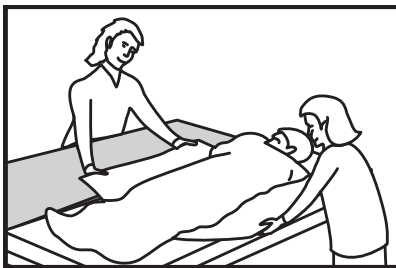

Using the draw sheet, slide the patient over the Half Shifter onto the stretcher.

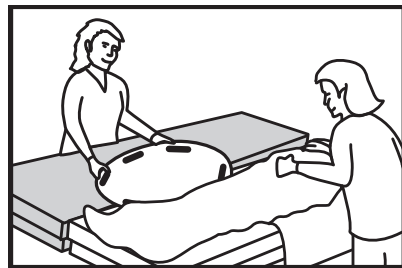

Remove the Half Shifter from beneath the patient.

CAUTION: *This is not a spine or backboard. Use extreme care to avoid neck and back injuries.*

Made in USA for:



AliMed® inc.

Dedham, MA 02026-9135

www.AliMed.com

800-225-2610

©2008 AliMed® inc. All rights reserved. #5802