




ChairProTM LIFT-OFF LAP CUSHION ALARM

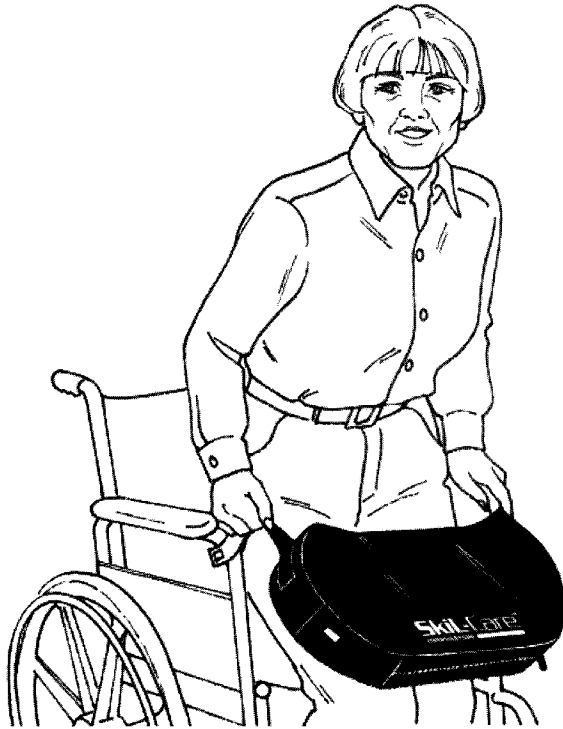

| Model # | Fits Wheelchairs | Unit |
|---------|------------------|------|
| 909374 | 16"-18" | Each |

This effective new alarm system combines a patient positioning lap cushion with a chair-exit alarm. An integral magnetic switch activates an alarm when the patient releases the Velcro® closure either by hand or by rising from the wheelchair. Skil-Care's Chair-Pro Lift-Off Lap Cushion Alarm as non-restrictive and may be used as part of a restraint-reduction program. Replacement guaranteed for one full year, this alarm system eliminates the cost of having to buy expensive, frequently replaced sensor strips. It may be used with 16-inch- to 18-inch-wide wheelchairs.

PURPOSE


Skil-Care ChairPro Lift-Off Lap Cushion Alarm alerts caregivers when a patient is about to leave, has left, or has fallen from the wheelchair.

CAUTIONS

1. The ChairPro Left-Off Lap Cushion Alarm is not a substitute for proper supervision. Fall-risk wander-risk patients should be monitored regularly.
2. Check battery each time alarm is used. If weak, replace the 9-volt battery.
3. LED should flash when patient/resident is seated and the magnetic switch is engaged. If LED fails to flash, check battery.

Alarm unit made in China. Instructions printed in USA.


INSTALLATION - ALARM UNIT

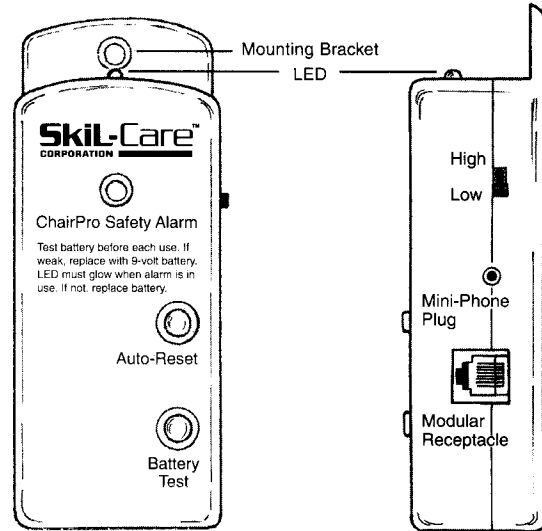

You may choose one of the following two ways to attach the alarm unit to the wheelchair.


Mounting Bracket

1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest.

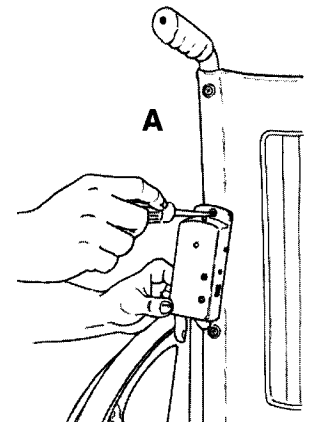


2. Remove grommet from screw, place the screw through hole on alarm unit mounting bracket, and fasten screw to wheelchair backrest. (Illustration A)

Cord Loop

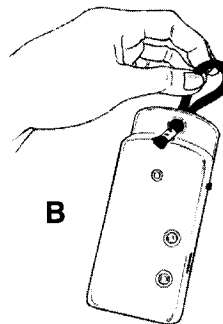

1. Insert enclosed cord loop through bracket hole. (Illustration B)

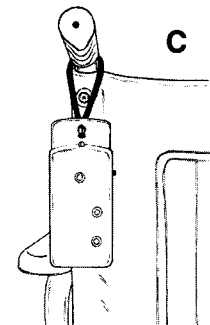


2. Hang from wheelchair push-handle (Illustration C)

Continued on reverse side.


INSTALLATION AND OPERATION- LIFT-OFF LAP CUSHION

1. Remove battery compartment cover and install
2. Press Battery Test button. If alarm doesn't sound, check battery connection and retest. If alarm fails to sound, replace battery.


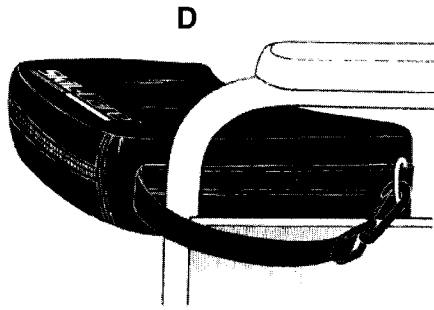


3. Seat the patient in the wheelchair and place the cushion on his/her lap (logo side up and zipper facing front). Bring the web strap with clip around the armrest support bar and secure the clip to the D-ring. (Illustration D)

4. Insert telephone-style plug into Modular Receptacle on right side of the alarm unit. (Illustration E) Alarm will sound. Press Auto-Reset button to silence alarm.

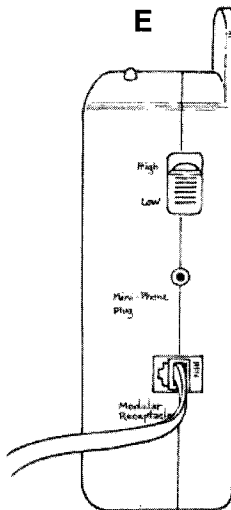

5. Bring strap with red loop around wheelchair armrest support bar, place magnet on metal strip, and secure Velcro® closure. (Illustration F) The LED will flash indication that the alarm is armed. Test the system by pulling magnet away from metal strip. The alarm will sound. Replace magnet on metal strip to silence and reset alarm.

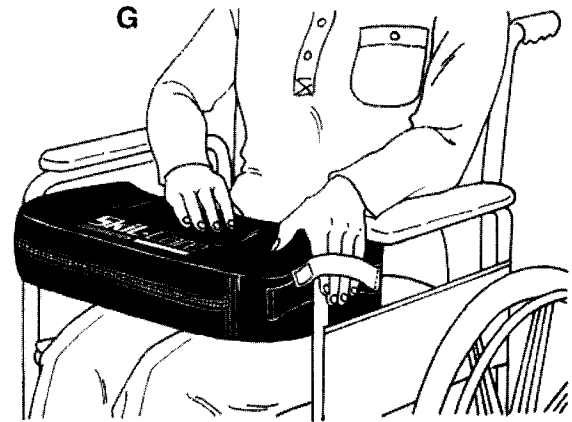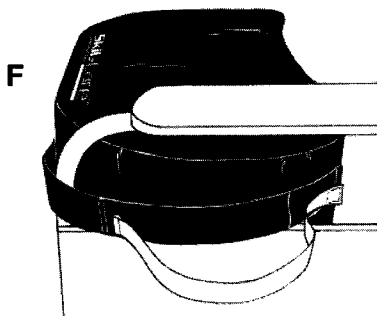

6. Select high or low alarm volume by means of the High/Low switch on the right side of alarm unit.

7. The alarm will activate if the patient releases the strap by hand (Illustration G) or stands up. Silence the alarm by pressing the Auto-Reset button. This places the alarm in the stand-by-mode. The alarm is automatically rearmed when the magnet is again placed on the metal strip. **There is no on/off switch.**

Important: Do not secure the magnet to the metal strip when the alarm is not being used to monitor your patient. Doing so will decrease battery life.

SAFETY REMINDERS

1. Check battery before each use. If alarm is weak or fails to sound, replace 9-volt battery.
2. When alarm is in use, check LED regularly. If it is not flashing, replace 9-volt battery. If LED fails to flash after battery has been replaced, make certain that the magnet is properly secured to metal strip.

GUARANTEE

The alarm unit and the lap cushion with magnetic switch are replacement guaranteed against defects in materials and workmanship under conditions of normal use for a period of one year from date of purchase. Please return defective part to dealer from whom the system was purchased. The dealer will send the defective part to Skil-Care and arrange for shipment of a replacement. This guarantee does not cover the battery supplied with the alarm unit.

Skil-CareTM
CORPORATION

*Advanced design for nursing and therapy.*TM

Alarm unit made in China. Instructions printed in USA.

Visit us online
at Skil-Care.com