Rolyan® Elbow/Heel Protector with Shock-Absorbing Pad: A730-001 through A730-003

Rolyan® Elbow/Heel Protector: A731-S through A731-XL

Rolyan® Workhard® Elbow Protector: A731-10 through A731-14

Rolyan® Neoprene Elbow Sleeve: A950-1 through A950-5

Be sure to note which product you are using and follow the instructions that apply to that product. Unless otherwise stated, these instructions apply to all of the products listed.

INDICATIONS

- To help protect the elbow

Rolyan Elbow/Heel Protector with Shock-Absorbing Pad

Rolyan Elbow/Heel Protector

- Can also be used to help protect the heel

CONTRAINDICATIONS

- Not for use over open wounds and infected areas
- Not for immediate treatment following fractures
- May not be for people with circulatory problems

Rolyan Workhard Elbow Protector

Rolyan Neoprene Elbow Sleeve

- Not for people who have an allergic reaction to neoprene or who have a tendency to develop dermatitis

PRECAUTIONS

- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the protector is not so tight that it interferes with circulation.**

CARE AND CLEANING

- If protector has hook-and-loop fasteners, close them before washing.
- Hand wash in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the protector is completely dry before reapplying it.

Rolyan Elbow/Heel Protector with Shock-Absorbing Pad

- Remove padding before washing protector.
- Pad can be washed with disinfectant or soap and water and wiped dry.
- Sleeve, machine wash in warm water and tumble dry with low heat.

Rolyan Workhard Elbow Protector

Rolyan Neoprene Elbow Sleeve

- Wash frequently to remove body oils and other soil.
- Do not use ointments or oils under protector.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

INSTRUCTIONS FOR USE

If protector has straps, open them. Put protector onto extremity, centering joint in contoured area or over padding.

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Patterson Medical, Ltd.