

IMAK ADJUSTABLE PIL-O-SPLINT

PATTERSON PART NUMBER:

081192939

Instructions for Use:

1. With the white lining facing up, place the palm of your hand over the cushioned pad on the Pil-O-Splint.
2. The thumb should protrude slightly through one of the side holes. (The hole on the opposite side is for using the splint on the other hand. Do not put your little finger through the second hole!)
3. Roll the straight side snugly over your hand and forearm.
4. Wrap the hook and loop strap diagonally over the Pil-O-Splint. Adjust to desired pressure. Once adjusted to the perfect fit, simply slide the Pil-O-Splint on and off without opening the hook and loop strap.
5. initially, wear the Pil-O-Splint for up to two hours. As you adjust to it, you may wear it to sleep throughout the night. If you are wearing the Pil-O-Splint as a result of surgery or other serious injury, please be sure to follow your doctor's advice carefully.

Care Instructions:

Remove splint, machine wash in cold water; tumble dry cool.