

HAND WALKER CUFF, PAIR

9265-00

Decide where would be best for user's hand placement on walker.
Once you have decided, place a piece of the provided white Velcro loop with adhesive backing into position wrapping it around the hand grip. Cut to reduce length, if necessary.

The patient can then put the palm guard securely on and grasp the walker handle.

The Velcro hook on the cuff will enable it to affix firmly to the Velcro loop on the walker.

Cuffs can be left in place on walker when not in use.

*2" W x 5" L
Loop
Attaches to
Walker*

SAMMONS
PRESTON
Rolyan

1-800-323-5547

Fax: 1-800-547-4333

