
A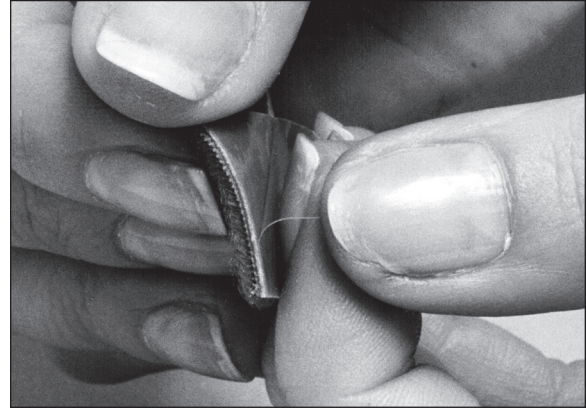

B

INDICATIONS

- For use with traction device for dynamic splinting

CONTRAINDICATIONS

- Not for use when sensation is absent or significantly impaired
- May be contraindicated for people with circulatory problems

PRECAUTIONS

- **These finger hooks are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**

- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **Contact with perspiration or water may reduce the tackiness of the slip-resistant surface; it should be checked frequently for adequate slip resistance (see Care and Cleaning for restoring tackiness).**

CARE AND CLEANING

Wash slip-resistant surface with mild soap to restore tackiness.

PROCEDURE

Leave protective covering on the slip-resistant surface until all adjustments have been completed.

1. With the protected slip-resistant surface against the skin and the dress hook away from the skin, center the dress hook, open end distal, on the bed of the nail.
2. Gently wrap the pile fabric around the finger and fasten the long hook strap to it (A).
3. Check the fit and trim the material as needed to reduce bulkiness and provide a snug fit. All edges can be trimmed, but avoid trimming close to the dress hook, and leave enough of the long hook material to provide a secure attachment.
4. Remove the protective covering from the slip-resistant surface before applying the wrap-on finger hook (B).

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Patterson Medical, Ltd.