

IMAK UNIVERSAL PIL-O-SPLINT

PATTERSON PART NUMBER:

081192921

Instructions for Use:

1. Slip your affected hand into the Pil-O-Splint so the palm of your hand rests on the cushioned pad. The rigid side should be facing up.
2. The thumb should protrude slightly through one of the side holes. (The hole on the opposite side is for using the splint on the other hand. Do not put your little finger through the second hole!)
3. Initially, wear the Pil-O-Splint for up to two hours. When you adjust to it, you may wear it to sleep throughout the night.

Care Instructions:

Remove splint, machine wash in cold water; tumble dry cool.