

## Homecraft Shower Chair with Back

0815-61-893



### Before Assembly

Inspect components for any damage that may have happened during shipping or handling. In the case of damage, contact Patterson Medical immediately.

**Please read these instructions before attempting assembly. If they are not clear, contact a healthcare professional or technician.**

### Assembly Instructions

Start by setting the chair upside down on an even, secure surface. After following steps above, attach the back to the base by pressing buttons on support tubes and inserting them into the support brackets on the frame. You will hear two "snaps" when the two pieces are fastened securely.

### Adjusting Height of Chair

Choose the desired height between 16 $\frac{1}{2}$ " and 22" H in  $\frac{1}{2}$ " increments. Set the height by pushing the snap button and then moving the leg extension up or down to match the appropriate hole. Release the

snap button until it "clicks," indicating that the button is fully engaged in the hole. Repeat on each leg, making sure the height of hole is the same as the first.

### Care

Clean with water and a mild soap or non-abrasive cleaning product. Test the cleaner on a hidden area such as the bottom of chair; this will show whether the cleaner would cause stains or discoloration.

Before every use, check the button/hole connection and that all four legs are even in height and secure in the shower/tub.

### Precautions

Do not exceed the maximum user weight capacity of 315 lbs. Always inspect the shower chair to ensure it is properly positioned and stable before using. Do not use if chair is wobbly or unstable. On a regular basis, inspect rubber tips to determine if they are ripped or cracked. In the case of wear, replace tips immediately.

**Ensure that all push buttons are locked in place before use.**

### Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



### Patterson Medical

1000 Remington Blvd., Suite 210  
Bolingbrook, IL 60440  
Phone: 800-323-5547

[www.pattersonmedical.com](http://www.pattersonmedical.com)

© July 2011 Patterson Medical Products, Inc.  
081561893LIT

Direct Phone: 630-378-6000  
Fax: 800-547-4333  
Direct Fax: 630-378-6310



### Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK

United Kingdom  
Phone: 08444 124 330  
Fax: 08448 730 100  
International  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784