



# TR<sup>2</sup> Tamper-Resistant Alarm, #71 0985

## Instructions for Use, Application, and Care


**Warning:** Before using this TR<sup>2</sup> Alarm, read entire instruction sheet and save for future reference. Failure to do so may result in injury or death to a person in your care.

### Use

Your new AliMed<sup>®</sup> TR<sup>2</sup> Tamper-Resistant Alarm includes four AA batteries. This alarm control unit supports sensor devices including: bed and chair sensor pads, seatbelt alarms, wheelchair alarms, and seat cushion alarms.

### Application

- Battery installation: Open battery compartment by loosening screw on base of the cover. Install batteries accordingly. Ensure battery is lined up correctly
- Remove the temporary reset button identification sticker on the front of the unit
- Slide the switch at the top of the unit to the LOAD position
- Insert the sensor cable into the sensor slot that has been revealed by moving the slide switch
- Move the slide switch into the LOCK position—**Test alarm\***
- Molded reset buttons on each side must be pressed simultaneously to reset the alarm

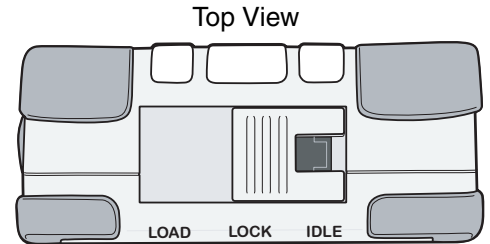

The alarm is now on and locked, and ready for use. The lock position incorporates a cable strain relief mechanism to reduce tensile strain on the cable and fitting. **NOTE 1:** Alarm cannot be turned off when in use. To store alarm, remove cable and slide switch to IDLE position to conserve batteries. **NOTE 2:** Test alarm with sensor device before using with patient to ensure both alarm and sensor are operating properly.

Test alarm daily to ensure patient safety.


### Customized Setup Options

Loosen the screw on the base of the case just enough to remove the case lid. Three setup options are available;

**Delay:** 0 or 2 seconds—Set to 0 seconds for immediate detection of movement from high fall risk patients. Set to 2 seconds to allow patients to readjust themselves in bed or chair.

**Alert:** chime, alarm, or silent. **Silent setting should only be used with the nurse call jack connection for use with nurse call systems.** This allows the alarm to sound only at nurse call station.

**Volume:** 85 dB or 100 dB—set to 85 dB for a quieter sounding alarm. Set to 100 dB for a louder sounding alarm if there are other noise-generating devices present.

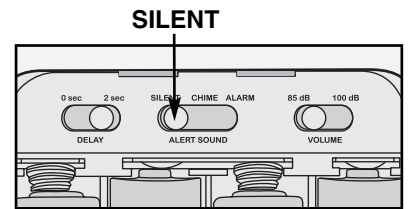

### Care

- Wipe clean using standard cleaning disinfectants
- Do not immerse in fluids
- Check function before each use. Do not use equipment that has been damaged

### Reorder Information

- #71 0985 Tamper-Resistant Alarm Control Unit
- #79366 Bed Sensor Pad Unit, Six-Month Warranty
- #79367 Bed Sensor Pad Unit, Six-Month Warranty, 10/cs
- #77061 Chair Sensor Pad Unit, Six-Month Warranty
- #77063 Chair Sensor Pad Unit, Six-Month Warranty, 10/cs



**STATUS** light will blink once when turned on. It will beep and blink when activated or reset.

**ALERT** light flashes continuously while alarm sounds.

**LO BATT** light indicates that batteries must be replaced if blinking.

Nurse call jack socket

Molded reset buttons (one on each side).

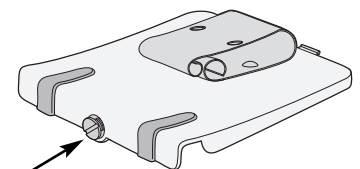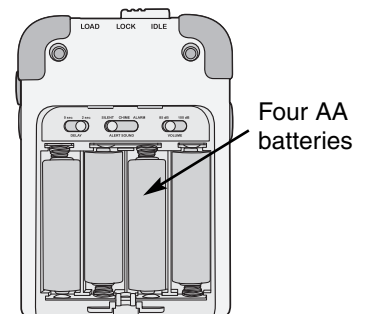

Captive screw won't get lost.

Distributed by:



Order Toll Free 800-225-2610 • 781-329-2900 • Fax 800-437-2966

#5957REV0910  
©2010 AliMed<sup>®</sup> inc. All rights reserved.