

# Folding SHOPPING CART



## Components List

<b>A</b>  New Cotter Pin (2)	<b>B</b>  Spring Cotter Pin (4)	<b>C</b>  Washer (12)
<b>D</b>  Large Spring (2)	<b>E</b>  Small Spring (2)	<b>F</b>  Friction Cap (2)
<b>G</b>  Front Wheel (2)	<b>H</b>  Rear Wheel (2)	<b>I</b>  Rear Axle (1)
<b>J</b>  Front Axle (1)	<b>K</b>  Pole Cap (2)	<b>L</b>  Preassembled Frame (1)

## Assembly Instructions

1. Flip the preassembled frame (L) upside down for easier assembly.
2. Place the components onto the front axle (J) as shown, Fig. 1.
3. Insert one end of the axle into the hole at the base of the preassembled frame (L) cart handle. Push in as far as possible and insert the other end of the axle into the other hole at the base of the cart handle, Fig. 3.
4. Place a washer (C) onto both ends of the front axle. Insert a spring cotter pin (B) into the holes on either end. Place the pole cap (K) over the end of each frame tubing, Fig. 4.
5. Insert the rear axle (I) through the holes at the bottom of the back of the preassembled frame (L). Secure in place by inserting the spring cotter pins (B) into the interior holes.
6. Place the components onto the rear axle (I) as shown, Fig. 2.
7. Insert the new cotter pin (A) through the hole on the rubber friction cap and axle, Fig. 5. Spread the new cotter pin (A) apart with pliers once inserted.

**Tools Needed:** Be sure that you have pliers before you begin assembly.

**Note:** If any parts are missing do not return to place of purchase. Contact Customer Care at 877-505-6190.

**Warning:** Do not use if any parts appear to be loose or damaged as personal injury may result.

DMI Shopping Cart is not a toy. Do not let children play with it or ride inside.

Weight capacity: 100 lbs.

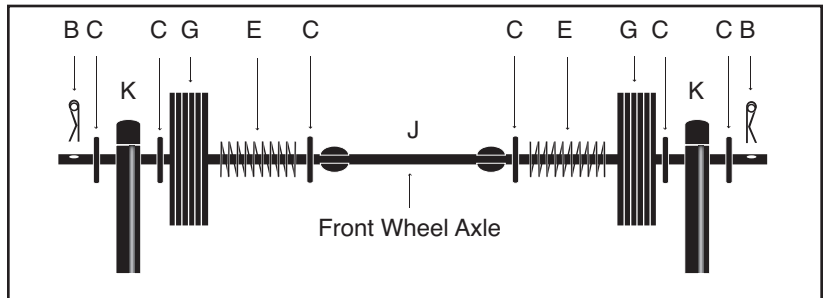


Fig. 1

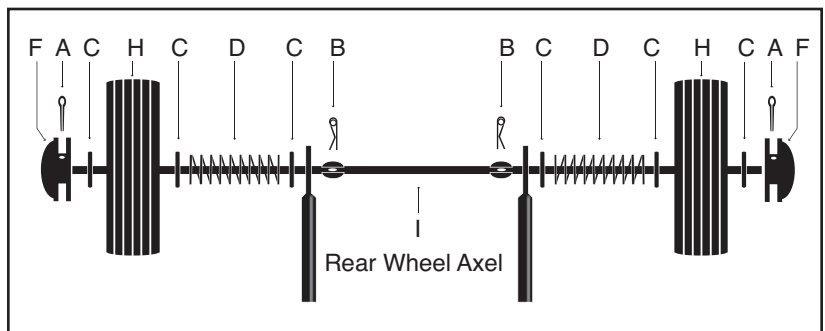


Fig. 2

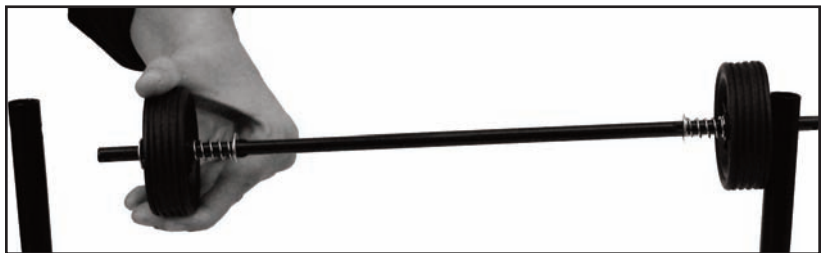


Fig. 3



Fig. 4



Fig. 5