Steam Inhaler Guidebook

Model 40-741-000

**Please read this guidebook
completely before operating this unit.**

**TOLL-FREE CUSTOMER CARE HELPLINE:
1-800-622-4714**

**Your physician or healthcare professional
can assist you in determining your
optimal treatment schedule.**

English • Spanish

Limited One-Year Warranty

MABIS Healthcare ("MABIS") guarantees that its MABIS Steam Inhaler will be free from manufacturing defects under normal use for a period of one year from the date of purchase.

This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the MABIS Steam Inhaler. This warranty is voided if the MABIS Steam Inhaler product is misused or abused in any manner.

If the MABIS Steam Inhaler fails to operate during the warranty period, return it postage prepaid along with \$5.00 for return shipping and insurance to: MABIS Healthcare, Attn: Repair Department, 1931 Norman Drive, Waukegan, IL 60085. Repair or replacement of the defective unit, at MABIS' option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations or how long an implied warranty lasts., so the above limitation may not apply to you.

This warranty constitutes MABIS' only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. **MABIS will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss**, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. MABIS shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

MABIS Healthcare
1931 Norman Drive
Waukegan, IL 60085

Index

Introduction	2
Product Identification	3
Cautions	3
Cleaning Procedures	4
Operating Your Inhaler	4-5
Troubleshooting	5
Product Specifications	6


Introduction

Thank you for purchasing a MABIS Steam Inhaler. With proper care and use, your Steam Inhaler will provide you with many years of therapeutic relief.

This unit operates on standard AC power. Treatments are delivered quickly, safely and conveniently with the use of tap water.

We encourage you to thoroughly read this guidebook to learn about the features of your Steam Inhaler.

Product Identification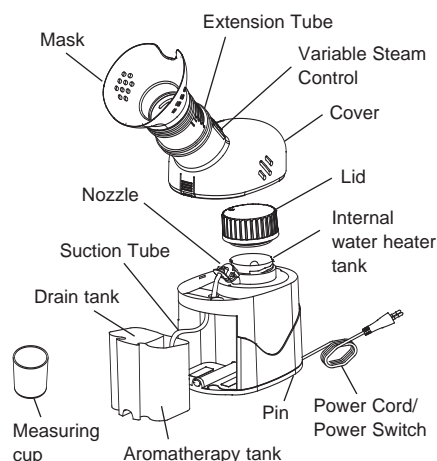

Cautions

Product Cautions

Please use general safety precautions when operating your Steam Inhaler.

This unit should be used only for (1) its intended purpose as described in this guidebook, and (2) with medications only under the supervision and instruction of your physician.

READ THE FOLLOWING BEFORE USING

- To avoid electrical shock: Keep unit away from water • Do not handle the unit or power cord with wet hands • Do not immerse the unit in liquid • Do not use while bathing • Do not reach for a unit that has fallen into water – immediately unplug the unit.
- Do not use the unit if it has any damaged parts (including power cord or plug), if it has been submersed in water or dropped. Promptly send the unit for examination and repair.
- Do not use extension cords.
- The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
- Keep the air vents open. Do not place the unit on a soft surface where the openings can be blocked.
- Do not operate the unit if the water tank is empty.
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.

- Never use without water in the water heater tank.
- The unit should not be left unattended while plugged in.
- Do not tilt or shake the unit when in operation.
- Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
- Do not use attachments unless recommended by the manufacturer.
- **Never add medicines or aromatherapy liquids to the water heater tank.**

Operating Cautions

- Close adult supervision is highly recommended when this unit is used by invalids or children.
- Keep your eyes away from the output of vapor mist.
- The maximum capacity of the water heater tank is 25 mL and should not be overfilled.
- The maximum capacity of the aromatherapy tank is 60 mL and should not be overfilled.
- Do not use this unit while operating a vehicle.
- If any discomfort or abnormality occurs, stop using the unit immediately.

Storage Cautions

- Do not store the unit in direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.
- Always keep the unit unplugged and empty while not in use.

Cleaning Cautions

- Check the mask, extension tube, tank, suction tube and any other optional components before each use. Dirty or worn parts should be replaced.
- Do not immerse the unit in water. It may damage the unit.
- Disconnect the unit from the electrical outlet before cleaning.
- Clean all necessary parts after each use as instructed in this guidebook.


Cleaning Procedures

Rinsing Technique


(Performed after each use):

- 1) Make sure that the unit has cooled (approx. 10 minutes), the POWER switch has been turned to the 'OFF' position and the unit has been disconnected from the power source.

- 2) Remove the cover from the body of the Steam Inhaler. Remove the cap to the water heater tank, Fig. 1.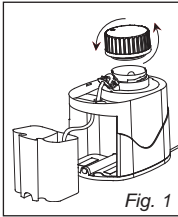

- 3) Carefully remove the one-piece aromatherapy/drain tank from the unit, Fig. 2.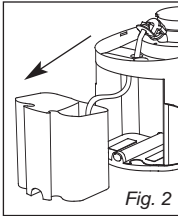

- 4) Rinse aromatherapy/drain tank with hot tap water.

- 5) Empty any remaining water from the water heater tank.

- 6) Dry with clean towels or allow to completely air dry.

- 7) Reassemble the Steam Inhaler and store in a cool, dry place.

Cleaning Technique

(Performed once a week or before first time use):

- 1) Follow steps 1-5 under 'Rinsing Technique.'
- 2) Wash aromatherapy/drain tank, cover, water heater tank cover, extension tube and mask in warm water and a mild detergent. Thoroughly rinse and allow to completely dry.

Cleaning the Steam Inhaler

NEVER submerge the main unit in water.

Wipe the Steam Inhaler using a soft, slightly water-moistened cloth.

NOTE: Any other form of cleaning or cleaning agents may damage the unit and void your warranty.

Operating Your Inhaler

Follow the cleaning instructions in this guidebook under 'Cleaning Technique' prior to using your Steam Inhaler for the first time or after it has been stored for an extended period of time.


REMEMBER: Always unplug the Inhaler and make sure the POWER switch is turned to the 'OFF' position before cleaning, assembling and before or after each use.

- 1) Place the Inhaler on a flat, stable surface within reach, with the mask facing towards you, Fig. 3.


- 2) Gently pull straight up on the cover of the Inhaler to remove lid.

- 3) Remove the cap to the water heater tank by turning it counterclockwise, Fig. 4.

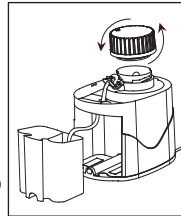
- 4) Using the included measuring cup, fill the water heater tank with approximately 25 mL of tap water, Fig. 5. Do not fill water above the five square-shaped vents on the wall of the tank. **NOTE:** Do not use anything except water in the water heater tank.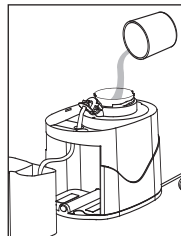

- 5) Reattach the cap and the cover.

- 6) If desired, remove the aromatherapy/drain tank from the unit. Fill the aromatherapy tank with up to 60 mL of saline or aromatherapy water, Fig. 6.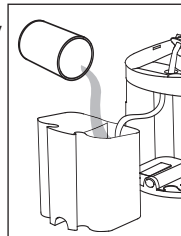

- 7) Return the aromatherapy/drain tank to the unit making sure that the suction tube makes contact with the bottom of the tank.

- 8) Position the extension tube to a comfortable position by manually extending or retracting the tube.
- 9) Slide the variable steam control down for small-sized particles of steam. Smaller particles are ideal when inhaling steam through the nose. Slide the control up and large-sized particles will result. Larger particles are ideal when inhaling through the mouth.

NOTE: Extending the tube also lowers the temperature of the steam.

- 10) Check that the POWER switch is in the 'off' position. Plug the power cord into a main outlet.

- 11) Turn the POWER switch to the 'ON' position.

- 12) Start your treatment.

- 13) The Steam Inhaler will begin to output steam in approximately 60 seconds. Once the level of steam becomes weak (approximately 6-9 minutes), turn the POWER switch to the 'OFF' position and unplug the power cord from the outlet.

IMPORTANT!

The Inhaler will automatically shut off if it becomes overheated. If/when this happens, immediately:


- 1) Turn the POWER switch to the 'OFF' position.
- 2) Unplug the power cord from the outlet.
- 3) Allow the unit to cool for approximately 30 minutes.

Before restarting the unit, make sure that the air vents on the extension tube are not obstructed.

After Every Use

- 1) Unplug the unit from the power source.
- 2) Allow the unit to completely cool (approximately 10 minutes).
- 3) Carefully remove the aromatherapy/drain tank and pour out any remaining water.
- 4) Follow the cleaning procedures provided in this guidebook.

NOTE: Occasionally the nozzle that directs the steam into the extension tube may become obstructed. If the Steam Inhaler begins to produce a significantly smaller amount of steam or no steam at all, the nozzle needs to be cleaned. To clean the nozzle, remove the cover from the Steam Inhaler. Firmly grasp the nozzle and twist clockwise to remove, Fig. 7. Insert a fine gauge needle or the pin which is stored under the bottom of the Steam Inhaler into the nozzle to clear the airway, Fig. 7. To reattach the nozzle, align the nozzle brackets with the grooves on the nozzle base. Turn counterclockwise until the nozzle locks into place.

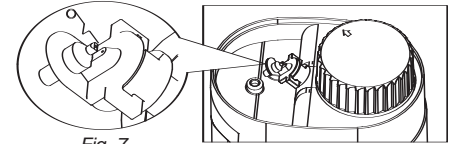Troubleshooting

If any abnormality occurs during use, please check and correct the following:

Condition	Correction
Unit does not operate when POWER switch is turned.	Check the AC connection to the outlet.
No steam or low rate of steam	Check that there is water in the water heater tank. Check that there is saline or aromatherapy water in the aromatherapy/drain tank (if applicable). Make sure that suction tube and other components are properly attached. Make sure that the steam nozzle is clean. Remove nozzle and clear the airway with the pin or fine gauge needle.

Product Specifications

Name and Model Number	MABIS Steam Inhaler, #40-741-000
Power Consumption	Max. 300 Watt
Internal Water Heater Tank Capacity	25 mL
Aromatherapy Tank Capacity	60 mL
Water Boiling Time	Approximately 60 seconds
Operation Time	Approximately 6-9 minutes
Steam Temperature (Ideal)	109.0°F (42.8°C)
Measuring Cup Capacity	25 mL
Dimensions	5-1/8" x 3-3/4" x 10"
Weight	1 lb. 5 oz.

Specifications are subject to change without notice.

 With respect to electric shock, fire and mechanical hazards only in accordance with UL 2601-1.

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