


CALF PAD COVER

Reorder #	Unit
703068	Pair

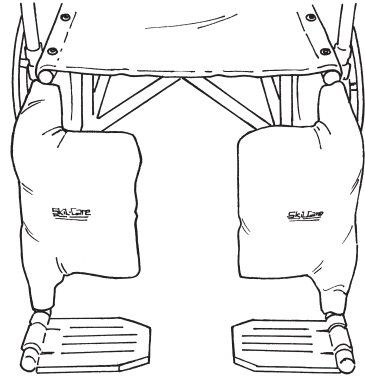

Purpose


Skil-Care's Calf Pad Cover is designed to reduce pressure on calves and protect the lower leg against skin tears and abrasions caused by contact with the wheelchair leg rest support bars.

Application


Note: The pads are sold in pairs and are labeled "left" and "right".

1. Slide wheelchair calf pad into pad cover sleeve and secure Velcro® backstrap. (Illustration A)
2. Wrap side of calf pad cover around the top and bottom of wheelchair foot support bar and secure Velcro® closures. (Illustration B)

A

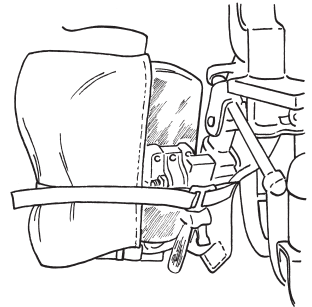

B

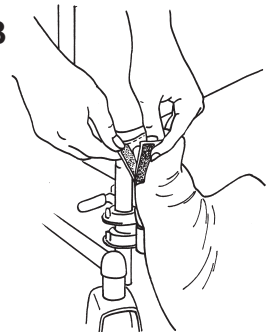

Easy Care

The Calf Pad Covers are covered with Skil-Care's Low-Shear fabric that wipes clean with a mild spray-type cleanser. The cover may be disinfected with a non-chlorine disinfectant. Do not launder.