Kings Cutlery Assessment Kit in Case AA5548

Description

This attractive and comprehensive range allows over 400 individual cutlery items to be made from 32 modular items. It caters for virtually all needs, economically and elegantly.

Fitting and Adjustment

1. **Basic Fitting - Utensil to handle** (See fig 1).
To assemble a cutlery utensil to any of the handles simply push the handle firmly on to the cutlery utensil until it becomes flush with the handle. Rotate the handle to the desired position by twisting gently.
2. **Use of the adaptor** (See fig 2).
If the angled adaptor is required, push the adaptor on to the cutlery utensil and then push the adaptor into the chosen handle. Rotate the handle to the desired position by twisting gently.
3. **Use of Double Ended Handles** (See fig 3).
A. If using a handle with two utensils, place them into the handle, following the basic fitting instructions. If only one utensil is used then the spigot provided may be placed in the other end to close off the opening.
B. If the strap is required with a single or double piece of cutlery, push the main cutlery item through the single hole end of the strap and place the other cutlery piece or spigot provided through the appropriate hole in the strap. Note the strap will stretch but must not be too tight. (See fig 4).
4. **Use of Vari-grip PVC Sleeves** (See fig 5).
To use the vari-grip built-up sleeves AA5545 etc, they must all be fitted over the AA5543 slim handle.

First attach the slim handle to the cutlery utensil as in the basic fitting instructions. Then slide the chosen sleeve over the open end of the handle and push it on until it is flush with the rim of the handle. Rotate the handle to the desired position by twisting gently.

5. Use of the Customised Handle Kit

One packet of FIMO is sufficient to make one handle. Work a small piece (a 1 cm / 1/4" block) in your hand until pliable; gradually add the rest of the block until it is all pliable. If you find this difficult, the FIMO can be placed on a radiator for 1 to 2 minutes to soften.

Form the shape required for the individual's hand and push the pinched end of the metal inner core into the FIMO leaving the rim of the open end protruding slightly.

Take care not to get any material in the open end and check that the moulded shape has not been distorted.

Harden in a standard oven at a maximum temperature of 130°C / 265°F for 20 to 30 minutes. Note: do not fit the cutlery utensil until it has been hardened and is cool.

To Disassemble

If you wish to change cutlery utensils or handles simply twist the handle gently and pull out.

Cleaning and Care

Most items in the range can be washed in the usual manner or in a dishwasher. The lightweight handles, however, should not be washed in a dishwasher.

Fig. 1

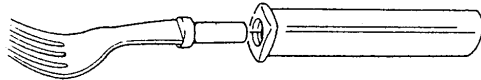
Fig. 2

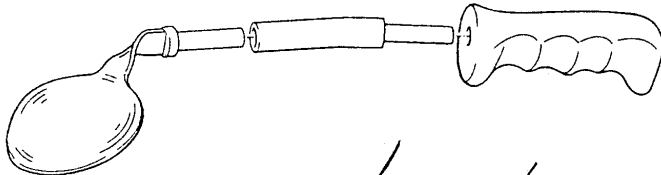
Fig. 3

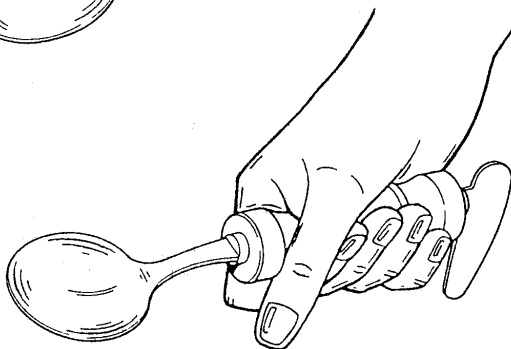
Fig. 4

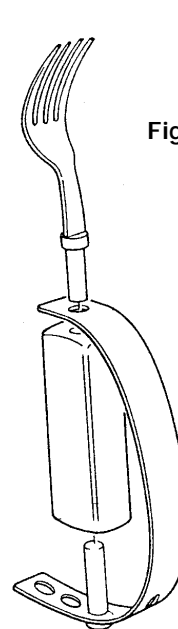
Fig. 5

Homecraft AbilityOne

Homecraft AbilityOne

Shelley Close, Lowmoor Business Park,
Kirkby-in-Ashfield,
Nottinghamshire, NG17 7ET, UK
Telephone: +44 (0)1623 754047 Facsimile: +44 (0)1623 755585

Sammons Preston Roylan

4 Sammons Court,
Bolingbrook,
IL 60440-4989, USA
Telephone: 1-630-226-0000 Telefax: 1-630-226-1390

Top Layer

- 1 AA5518 Kings Soft Coated Spoon
- 2 AA5519 Kings Soft Coated Teaspoon
- 3 AA5532L Kings Angled Spoon - Left handed
- 4 AA5532R Kings Angled Spoon - Right handed
- 5 AA5522 Kings Standard Spoon
- 6 AA5524 Kings Standard Teaspoon
- 7 AA5523 Kings Junior Spoon
- 8,9 AA5526 Kings Sipping Spoon (2)
- 10 AA5533 Kings Large Rocker Knife
- 11 AA5529 Kings Small Rocker Knife
- 12 AA5530 Kings Angled Knife
- 13 AA5528 Kings Nelson Knife
- 14 AA5520 Kings Standard Knife
- 15 AA5521 Kings Standard Fork
- 16 AA5531L Kings Angled Fork - Left handed
- 17 AA5531R Kings Angled Fork - Right handed
- 18 AA5527 Kings Splayed

Bottom Layer

- 1 AA5544 Kings Mushroom Handle
- 2 AA5542 Kings Customised Handle (Metal inner core and modelling material)
- 3 AA5547 Kings Vari-Grip Sleeve - Large
- 4 AA5546 Kings Vari-Grip Sleeve - Small
- 5 AA5545 Kings Vari-Grip Sleeve - Oval
- 6 AA5538 Kings Lightweight Foam Handle - Small
- 7 AA5537 Kings Lightweight Foam Handle - Large
- 8 AA5541 Kings Lightweight Double Ended Handle with Straps
- 9 AA5534 Kings Angled Adapter
- 10 AA5535 Kings Contoured Handle
- 11 AA5536 Kings Standard Handle
- 12 AA5539 Kings Heavyweight Handle
- 13 AA5543 Kings Slim Handle
- 14 AA5540 Kings Standard Double Ended Handle with Strap

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