

Description

This folding, height adjustable steel trolley has moulded plastic trays and angled handles to give improved grip and offer stability while moving food etc. The shelves have an open side for easier transfer of plates and dishes. The rubber castors give control on any floor surface.

Instructions for Use

1. To unfold, open the trolley frame, lift the plastic trays up, then clip onto the lip on the cross piece of the frame.
2. Insert the four castor legs into the tubes, adjustment holes facing downwards.
3. Depress the pin clips on the castor legs and push the legs into the tubes to the desired height. Ensure that the pin clips protrude through the required hole to lock the leg in position. Ensure all legs are set at the same height.

Specification

- Height Adjustable from 875 to 1060mm (34½" to 41¾").
- External depth 470mm (18½").
- Size when folded 480 × 185mm (18 ¾ × 7 1/2").

Cleaning and Care

All parts may be cleaned using a brush or a cloth and a solution of mild detergent or disinfectant.



These user instructions are available in large print, please contact Performance Health Customer Services for a copy.
Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

**PERFORMANCE
HEALTH**