

homecraft Wall Mounted Shower Seat with Legs



NEW CODE	OLD CODE	FRAME
09 117 1644	538DL	Aluminium
09 118 6691	538D	Steel

Introduction

The Homecraft Wall Mounted Shower Seat with Legs are manufactured from lightweight aluminium with a white powder coating. The seat is a HD polyethylene moulding with drain holes. The 538DL has seat support rails hinged at the plate allowing it to be folded away and locked into an upright position. It also has height adjustable legs which are required when extra support is needed.

Directions for use

The mounting plate has 8mm Diameter mounting holes to secure it to the wall. It should be fitted by a competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user. The fixing method will depend on the constitution and thickness of the wall and therefore we supply no wall fixings with the seat.

Leg adjustment

To adjust the height, grasp the 'snap lock' between forefinger and thumb and pull it out. Slide the leg out to the required position and push the clip firmly back into the hole. NOTE: Set the leg and rail height before securing the mounting plate to the wall.

Maintenance

The Homecraft Wall Mounted Shower Seat with Legs is designed for minimum maintenance and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives or bleach based cleaners which may cause damage or discolouration. Periodically check the wall plate fittings and the pivot point of the rail are secure.

Warranty Information

Performance Health warrants that the Homecraft Wall Mounted Shower Seat with Legs are free from defects in material and workmanship. The warranty shall remain in effect for 1 year from the date of original consumer purchase.

The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement.

This warranty does not cover the following:

- Replacement parts supplied by anyone other than Performance Health or the stockist;
- Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use;
- Performance Health is not liable for incidental or consequential injury or damage to property;

Should it be necessary to return the Homecraft Wall Mounted Shower Seat with Legs, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise Performance Health cannot accept any responsibility for transit damage to the unit.

This does not affect your statutory rights.



Maximum user weight **160 kg** **25 st**

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

**PERFORMANCE
HEALTH**