



Powerstep®



SLIM-TECH 3/4 LENGTH

MATERIALS


- Polypropylene Support Shell
- Polyester fabric top cover with anti-microbial protection, Royal Blue
- HyPUR-cel® Top Layer Charcoal Grey, 1/8"
- Non-Slip Pad at Heel to help hold orthotic in place

FEATURES & BENEFITS

- Strong, medical-grade foot support calibrated with flexibility
- Ultra slim design works in a wide range of shoe types
- 3/4 length with thin tapered edge
- HyPUR-cel® cushion top layer with slim profile
- Friction reducing, anti-microbial top fabric
- Low, flat heel cradle to stabilize and protect the heel during landing

INDICATIONS: Over-Pronation; Plantar Fasciitis/Heel Pain; Foot, Arch and Heel Pain; Calluses; Hammertoes and Claw Toe; Bunions; Neuromas; Ankle and Knee Pain

IDEAL SHOE TYPE: Dress and Casual Shoes, Slip-On Dress Shoes, Low to Mid Heel Dress Shoes, Athletic Cleats

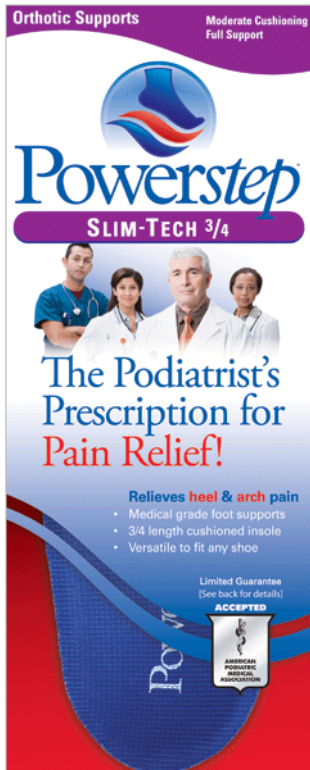

Label Code	Product Code	Size	Men's Size	Women's Size	UPC Code
ST-a	a2s	Asm	4 - 4 ½	6 - 6 ½	749852100123
ST-A	A2	A	5 - 6 ½	7 - 8 ½	749852100222
ST-B	B2	B	7 - 8 ½	9 - 10 ½	749852100321
ST-C	C2	C	9 - 9 ½	11 - 11 ½	749852100420
ST-D	D2	D	10 - 10 ½	12 - 12 ½	749852100529
ST-E	E2	E	11 - 11 ½	13 - 13 ½	749852100628
ST-F	F2	F	12 - 13	14 - 15	749852100727

WEARING/FITTING DIRECTIONS: Insert Powerstep 3/4 insoles into your shoes. If necessary, wear a few hours a day until feet are accustomed to better support, and then wear full time. Suggest wearing Powerstep insoles in all your shoes, even slippers, to give continuous foot support.

CARE INSTRUCTIONS: Spot clean with a mild detergent. Do not immerse in water. Air dry only. Replace your Powerstep insoles as needed depending on wear.

WARNING: Not intended for persons with severe foot deformities or circulatory disorders resulting in infections or foot ulcerations. Consult your podiatrist or medical professional before using. Always consult a medical professional when experiencing persistent pain, discomfort or swelling.



888.237.3668 · 513.825.1888
 Email: info@powersteps.com
www.powersteps.com

Stable Step, Cincinnati, OH
 Assembled with pride in the USA!