



Powerstep®



PINNACLE™ FULL LENGTH

MATERIALS

- Polypropylene Support Shell
- EVA Base Royal Blue
- Polyester fabric top cover with anti-microbial protection, Royal Blue
- HyPUR-cel® Top Layer Black, 1/8"

FEATURES & BENEFITS

- Strong, medical-grade foot support calibrated with flexibility
- Double layer, shock absorbing EVA/HyPUR-cel® cushion casing for enhanced comfort
- Friction reducing anti-microbial top fabric

INDICATIONS: Over-Pronation (Mild to Moderate); Plantar Fasciitis/Heel Pain; Foot, Arch and Heel Pain; Calluses; Hammertoes and Claw Toe; Bunions; Neuroma; Ankle and Knee Pain


IDEAL SHOE TYPE: Athletic Shoes, Dress and Casual Shoes

Label Code	Product Code	Size	Men's Size	Women's Size	UPC Code
PN-a	PNa1s	Asm	3 - 3 ½	5 - 5 ½	749852100154
PN-A	PNA1	A	4 - 4 ½	6 - 6 ½	749852100253
PN-B	PNB1	B	5 - 5 ½	7 - 7 ½	749852100352
PN-C	PNC1	C	6 - 6 ½	8 - 8 ½	749852100451
PN-D	PND1	D	7 - 7 ½	9 - 9 ½	749852100550
PN-E	PNE1	E	8 - 8 ½	10 - 10 ½	749852100659
PN-F	PNF1	F	9 - 9 ½	11 - 11 ½	749852100758
PN-G	PNG1	G	10 - 10 ½	12 - 12 ½	749852100857
PN-H	PNH1	H	11 - 11 ½	13 - 13 ½	749852100956
PN-I/J	PNI/J1	I/J	12 - 13 ½	14 - 15 ½	749852101052
PN-K	PNK1	K	14 - 14 ½		749852101151

WEARING/FITTING DIRECTIONS: Remove the shoe's existing insoles if possible and replace with your Powerstep insoles. Trim edges if needed to fit. If necessary, wear a few hours a day until feet are accustomed to better support, and then wear full time. Suggest wearing Powerstep insoles in all your shoes, even slippers, to give continuous foot support.

CARE INSTRUCTIONS: Spot clean with a mild detergent. Do not immerse in water. Air dry only. Replace your Powerstep insoles as needed depending on wear.

WARNING: Not intended for persons with severe foot deformities or circulatory disorders resulting in infections or foot ulcerations. Consult your podiatrist or medical professional before using. Always consult a medical professional when experiencing persistent pain, discomfort or swelling.

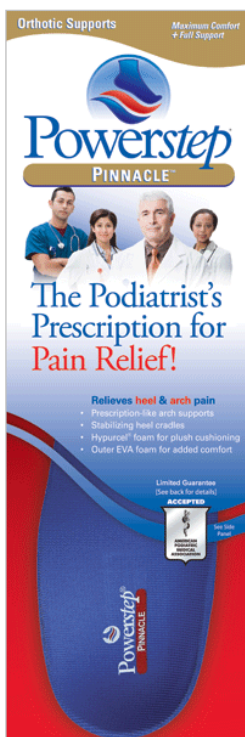

888.237.3668 · 513.825.1888
Email: info@powersteps.com
www.powersteps.com

Stable Step, Cincinnati, OH
Assembled with pride in the USA!