



ReOrder #	Description	Size	Units
915325	Gel-Foam Commode Cushion	15"W x 18.5"H	Ea.
909397	Replacement Straps	9" Long	6pk of 4

**Purpose**

The Gel-Foam Commode Cushion incorporates a top layer of gel and a bottom layer of foam to provide comfort and maximum pressure relief. The baffled gel pack design stabilizes the gel to insure that the gel doesn't migrate away from the high pressure areas. The heat-sealed vinyl casing is water proof and cleans easily with disinfectants and is soft and gentle on the skin. There are four Velcro straps that secure the cushion to the toilet seat.

**Easy To Use**

Place the Gel-Foam Commode Cushion on the toilet seat with the open end of the horseshoe in the front. Lift the toilet seat and secure the four Velcro straps (*Fig. 1*).

**Easy Care**

The vinyl seat cushion is heat-sealed and can be cleaned with a mild disinfectant. If a tear or rip is observed, the unit should be discarded. Over an extended time the liquid dissipates so the cushion should be replaced if the gel has disappeared.



*Fig. 1*

**Caution**

Although the gel is non-toxic, medical care should be consulted if the gel is ingested