


Posey® Over Mattress Bed Sensor #8307

(For Bed Use Only)

USE THIS SIDE UP

Warranted for 6 months of use.

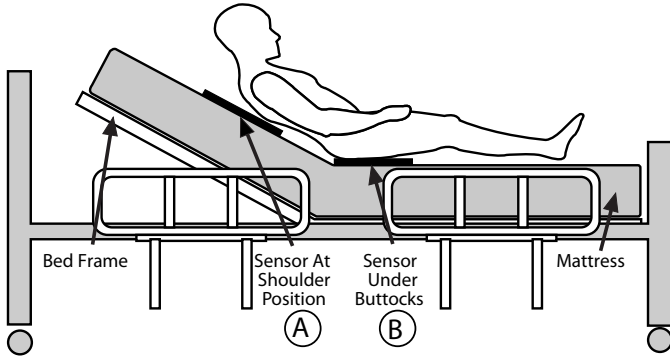

Note: This sensor is designed for use with Posey alarm monitors only.



5635 Peck Road • Arcadia, CA 91006-0020 USA
Tel: 800-447-6739 • Fax: 800-767-3933 • www.posey.com
MADE IN CHINA

Instructions

1. Enter sensor expiration date on the label provided below. Expiration is 6 months from first use.
2. Place the non-slip Posey Grip™ on the mattress.
3. Position sensor on top of mattress, across width of bed, in one of two positions (see illustration):
(A) Directly under patient's shoulders; alarm will sound when patient sits up.
(B) Directly under patient's buttocks; alarm will sound when patient moves off the sensor as in a bed exit.
4. Use the metal clips to secure the sensor to the mattress.
5. Place the linens on the bed **over** the sensor.
6. Connect the sensor to the Posey alarm unit.
7. Make sure the sensor cord is not stretched or stressed in any way and is clear from entanglement in the bed frame.

Always verify system is working properly before leaving patient unattended.

System Test:

1. Plug sensor into alarm unit.
2. Turn alarm unit on and verify power light is lit.
3. Check to make sure the audio alarm is functioning properly by lifting weight off the sensor pad and activating the alarm.

NOTE: A foam pad on top of the mattress may effectively diffuse the patient's weight so it does not activate the sensor. If the alarm does not properly activate, reposition the sensor above the foam pad.

4. Ensure the nurse call interface, when used, is connected properly and is activated properly.
5. Please refer to the alarm unit owner's manual for additional information.

Manufacture Date _____

⚠️ WARNING

To avoid possible injury or death:

- Read sensor and alarm instructions prior to use.
- This device may not be suitable for use with all "high fall risk" patients. Restraints may still be required.
- This device should not be a substitute for maintaining the routine visual monitoring protocol by healthcare personnel.
- Never connect Posey sensors to other manufacturers' alarm units.
- Always test system per instructions prior to use.
- Be sure restraint straps do not cross over sensor pad! Restraint straps may prevent alarm activation if patient falls off mattress.
- When using a restraint, always use side rails in the "up" position. The patient's body or limb should not fit over, under, around, through or between side rails.
- The alarm may fail to sound if the pad is bent or folded.

Warranty: This product is warranted for a period of up to 6 months. For longer duty life, we recommend use of the Posey Reusable Bed Sensor, #8285L, #8285H or #8292. See Posey Catalog for details.

Life of Product: We recommend this product be used for a period not to exceed 6 months. If, at any time, this sensor does not function during test prior to each use **stop use immediately and replace with a new sensor.**

Cleaning

Wipe clean with liquid disinfectant. **Do not submerge in liquid or sterilize with heat!**

For assistance, call 1-800-44-POSEY



L9311C 110606