

UNO WHO / UNO WHT

Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.




1. Position hand in brace.



4. Secure the straps.

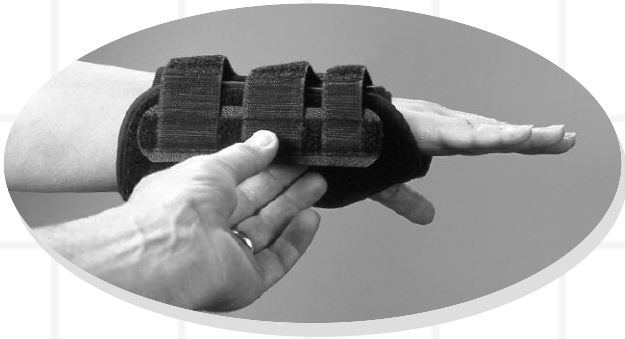

2. Move the stay pod into position along the side of the arm, below ulnar styloid process.

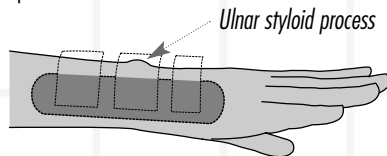

5. Trim excess strap material.



3. Pull straps through "D" ring.

UNO WHT



Additional step for UNO WHT
6. Secure thumb strap and trim extra strap material.

U.S. Patent No. 6960176

HELWY & WEBER

© 2007 Weber Orthopedic Inc.
DBA Hely & Weber All rights reserved.
1185 East Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE

EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

Rev A 11/08