# Tropic heater

by Sammons Preston

## User Manual

### Operation Instructions for:

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Tropic Heater 4-Pac

Tropic Heater 6-Pac

Tropic Heater 8-Pac

Tropic Heater 8-Pac Mobile

Tropic Heater 12-Pac Mobile

Tropic Heater 12-Pac Mobile without Pacs

ENGLISH / ESPAÑOL

# ENGLISH

Foreword..... 2  
Precautionary Definitions..... 2  
Safety Precautions..... 3-4  
Nomenclature..... 5-9  
Technical Specifications ..... 10-14  
Setup ..... 15  
Operation..... 15  
Troubleshooting..... 16  
Accessories..... 17  
Maintenance ..... 18-19  
Warranty ..... 20  
Replacement Parts ..... I-IX

Comet® of Prestige Brands, Inc.  
Soft Scrub® of the Dial Corporation  
Scotch-Brite™ of 3M

This manual contains general instructions for operation, precautionary instructions, and maintenance recommendations for the owners and operators of the Tropic Heater Heating Units. In order to obtain maximum life and efficiency of the Heating Units, please read this manual thoroughly and follow all instructions prior to operating the unit.

Specifications put forth in this manual were in effect at the time of publication. However, changes to these specifications may be made at any time without obligation on the part of Sammons Preston.

## PRECAUTIONARY DEFINITIONS

The precautionary instructions found in this section and throughout this manual are indicated by specific symbols. Understand these symbols and their definitions before operating this equipment. The definitions of these symbols are as follows:

= **CAUTIONS** - Text with a "CAUTION" indicator explains possible safety infractions that could have the potential to cause minor to moderate injury or damage to equipment.

= **WARNINGS** - Text with a "WARNING" indicator explains possible safety infractions that will potentially cause serious injury and equipment damage.

= **DANGERS** - Text with a "DANGER" indicator explains possible safety infractions that are imminently hazardous situations that would result in death or serious injury.

**NOTE:** Throughout this manual, "**NOTE**" may be found. These notes are helpful information to aid in the particular area or function being described.

## CAUTION

- Read, understand, and practice the precautionary and operating instructions. Know the limitations associated with the Tropic Heater Heating Units. Observe the cautionary and operational decals installed on the unit.
- DO NOT operate this unit in an environment where other devices are being used that intentionally radiates electromagnetic energy in an unshielded manner. Portable and mobile RF communications equipment can affect Medical Electrical Equipment.
- The Tropic Heater Heating Units generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning the Tropic Heater Heating Unit on and off. Try to correct the interference using one or more of the following: reorient or relocate the receiving device, increase the separation between the equipment, connect the equipment to an outlet on a different circuit from that which the other device(s) are connected and consult the Sammons Preston Service Department for help.
- Do not use accessories other than those supplied with the Tropic Heater Heating Unit or recommended by Sammons Preston. The safety of other products has not been established, and their use could result in injury to the patient.
- Never adjust the thermostat too high. The thermostat is extremely sensitive and the slightest adjustment will alter the temperature several degrees. The recommended operating temperature is 160° F to 165° F (71° to 74° C). The temperature of the water should be checked with a thermometer after every adjustment, before using the Tropic Pac. Always allow sufficient time for the water temperature to stabilize.
- The Tropic Heater Heating Unit should not be installed above a surface that is susceptible to casual water damage.
- Always keep the water level over the top of the Tropic Pacs to avoid burning or scorching of the packs.
- Never allow the Tropic Pacs to dry after use.
- Check water level daily as it has a natural loss due to evaporation.
- Clean the tank periodically as described in the maintenance portion of this manual.
- Check unit periodically for signs of leaking and contact service if leaks are noted.
- Always replace Tropic Pacs as soon as they show signs of wear.
- If the unit is to be left unattended for a period of time, unplug the unit, remove the packs, empty the water, and clean the tank.
- Do not move the Tropic Heater Heating Unit while filled with heated water. Tipping over could result in burns to the user or others.
- Always unplug the Tropic Heater Heating Unit from the power source before attempting to empty water from unit.
- Always use factory authorized replacement parts.
- Always destroy and replace Tropic Pacs which are exposed to contamination: hazardous or cleaning materials, bodily fluids, mold, etc.
- This unit should be operated, transported, and stored in temperatures between 0° F and 140° F (-18° C and 60° C) and atmospheric pressure range between 50 kPa - 160 kPa, with relative humidity ranging from 10% - 100%.
- Treatment time should not exceed 30 minutes.
- DO NOT retrieve Tropic Pacs from tank or replace Tropic Pacs in tank with unprotected hand.
- Avoid direct contact with mounting rack in tank.
- Always wrap Tropic Pac with towel or terry cover before handling or applying to patient.
- Constantly monitor Tropic Pac application to ensure that the skin is not becoming too hot.
- Damage to skin can occur from exposure to extreme heat or cold. Note instructions for proper use.
- DO NOT apply over insensitive skin or in the presence of poor circulation.
- Never lay or sit on top of the Tropic Pac.
- DO NOT use Tropic Pac directly over cuts, abrasions or wounds.
- Consult health care professional before using Tropic Pacs on non-communicative individuals.
- Exercise extreme caution when using Tropic Pacs on non-communicative individuals.

## **WARNING**

- **Grounding the Tropic Heater Heating Units:** Make certain the unit is electrically grounded by plugging into a hospital grade electrical outlet with a ground terminal receptacle (U-ground outlet). Follow the National Electrical Code.
- The water temperature in the Tropic Heater Heating Unit is approximately 160° F (71° C), and the water scalding temperature is approximately 120° F (49° C).
- Care must be taken when operating the Tropic Heater Heating Unit adjacent to or stacked with other equipment. Potential electromagnetic or other interference could occur to this or other equipment. Try to minimize this interference by not using other equipment in conjunction with it.
- The Tropic Heater Heating Unit should be placed on a smooth, level surface. The surface should support up to the Filled Weight (weight of heating unit filled with water) found in Technical Specifications.
- Use only accessories that are specially designed for the Tropic Heater Heating Unit. Do not use accessories manufactured by other companies on the Tropic Heater Heating Unit. Sammons Preston is not responsible for any consequence resulting from using products manufactured by other companies. The use of other accessories or cables may result in increased emissions or decreased immunity of the Tropic Heater Heating Unit.
- Periodically inspect unit for signs of damage and pitting during regular cleaning intervals, as described in maintenance portion of this manual. Contact service if any signs of damage are discovered.
- DO NOT apply Tropic Pac directly to skin.
- Individuals with circulatory problems should consult with a physician before using this product.
- DO NOT sit or lie on the Tropic Pac.
- DO NOT add chlorine additive to water.
- DO NOT allow Tropic Pac to come in contact with hydrocarbons (e.g., Comet®, Soft Scrub®, etc.).

Risk of explosion if used in the presence of flammable anesthetics.

## Explanation of Symbols

Refer to Instruction Manual/Booklet

Type B equipment

Thermostat Intensity Control

Caution! Hot Surface


## NOMENCLATURE

Tropic Heater Heating Units by Sammons Preston

**WHITE LAMP (where applicable)**, when illuminated, indicates that main power is connected, the switch is on and unit is operating.

**GREEN LAMP (220/240 volt units)**, when illuminated, indicates that main power is connected, the switch is on and unit is operating.

**RED LAMP (where applicable)**, when illuminated, warns of an overheat/low water condition. Action is necessary to correct the condition.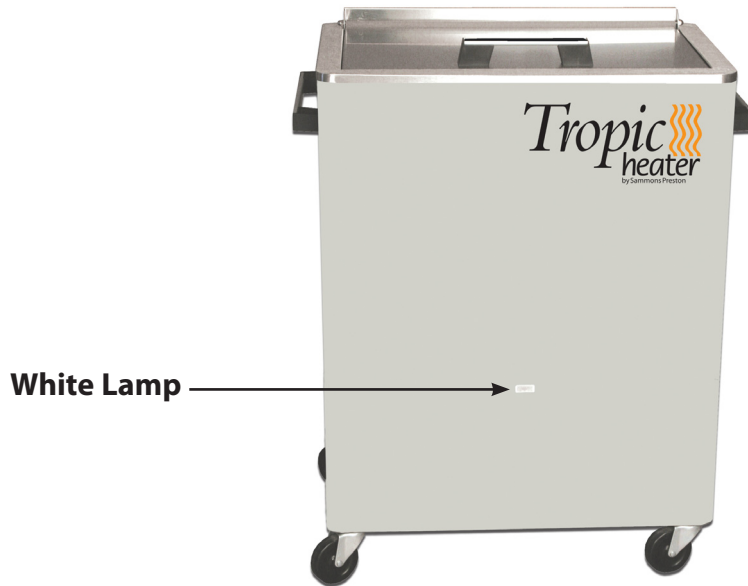

**IMPORTANT:** Some models are fitted with a thermal cut-out thermostat which acts as a "non self-resetting" thermal cut-out. In the unlikely event of a fault condition, this device is designed to open the power circuit between 190° F ± 14° F (88° C ± 8° C). It is not possible to alter its operating range. The operation of the unit may be restored by depressing the red button after the water/element temperatures have been allowed to cool. If the fault condition persists, please contact Sammons Preston or an authorized agent.

# NOMENCLATURE

Tropic Heater Heating Units by Sammons Preston

## Diagram


1. Lid
2. Lid Rod
3. Pilot Lamp, Clear
4. Power Cord
5. On/Off Switch
6. Thermostat Intensity Control
7. Drain Valve
8. Heating Element
9. Drain
10. Hydraulic Thermostat
11. Bracket (Bracket & Bulb Thermostat A/B Assembly)
12. Internal Wiring Harness
13. Drain, Copper Tube


**Tropic Heater 6-Pac**


**Tropic Heater 4-Pac (Rear View)**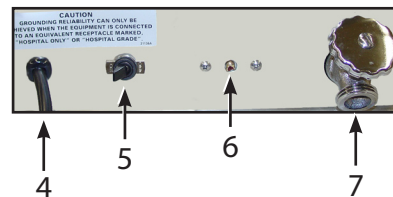

**Tropic Heater 6-Pac  
Rear View**

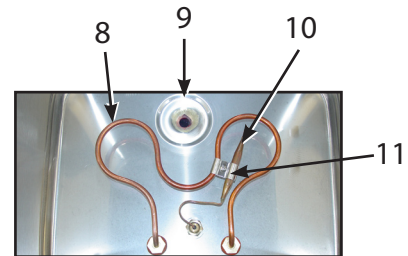**Tropic Heater 6-Pac  
Inside View**

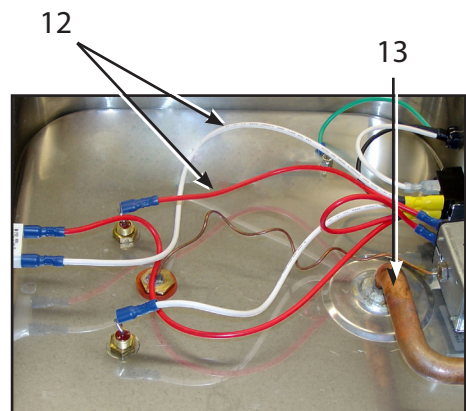
**Tropic Heater 6-Pac  
Underneath View  
(J-Box Cover Removed)**

**NOTE:** 4-Pac and 6-Pac 120 Volt models shown above; models will vary.

## Diagram (CONTINUED)

- 14. Lid Knob/Handle
- 15. Lid Rod Nuts
- 16. Side Handle
- 17. Casters, 3-inch
- 18. Neck Pak Insert
- 19. Tropic Pac
- 20. Junction Box (J-Box) Cover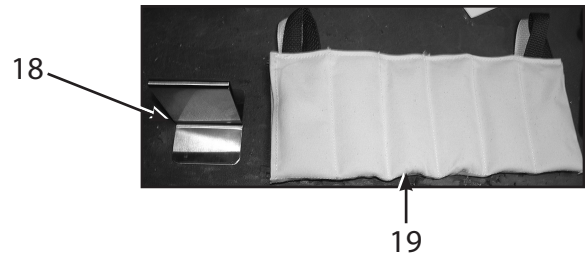

**Tropic Heater 12-Pac Mobile**

### **Not illustrated:**

- 21. Stainless Steel Wire Rack
- 22. Drain, Hose, Bib
- 23. Replacement Tank
- 24. Outer Jacket
- 25. Water Thief
- 26. Plastic Feet
- 27. Hose, Drain
- 28. Drain, Copper, Part B (Tropic Heater 8-Pac only)
- 29. Bracket, Element Support


**NOTE:** Tropic Heater 12-Pac Mobile 120 Volt model shown above; models will vary.

# NOMENCLATURE

Tropic Heater Heating Units by Sammons Preston

## Diagram

- |   |                               |
|---|-------------------------------|
| 1. Heating Element                                  | 9. Junction Box (J-Box) Cover |
| 2. Drain  | 10. IEC connector             |
| 3. Overtemp Thermostat                              | 11. Power Cord                |
| 4. Thermostat, Hydraulic                            | 12. Drain Valve (Not Shown)   |
| 5. Bracket (Bracket & Bulb Thermostat A/B Assembly) | 13. Reset Overtemp Button*    |
| 6. Lid  | 14. 6.3 Amp Fuses             |
| 7. 3-inch Caster                                    | 15. Pilot Lamp, Red           |
| 8. Overtemp Light                                   | 16. Pilot Lamp, Clear         |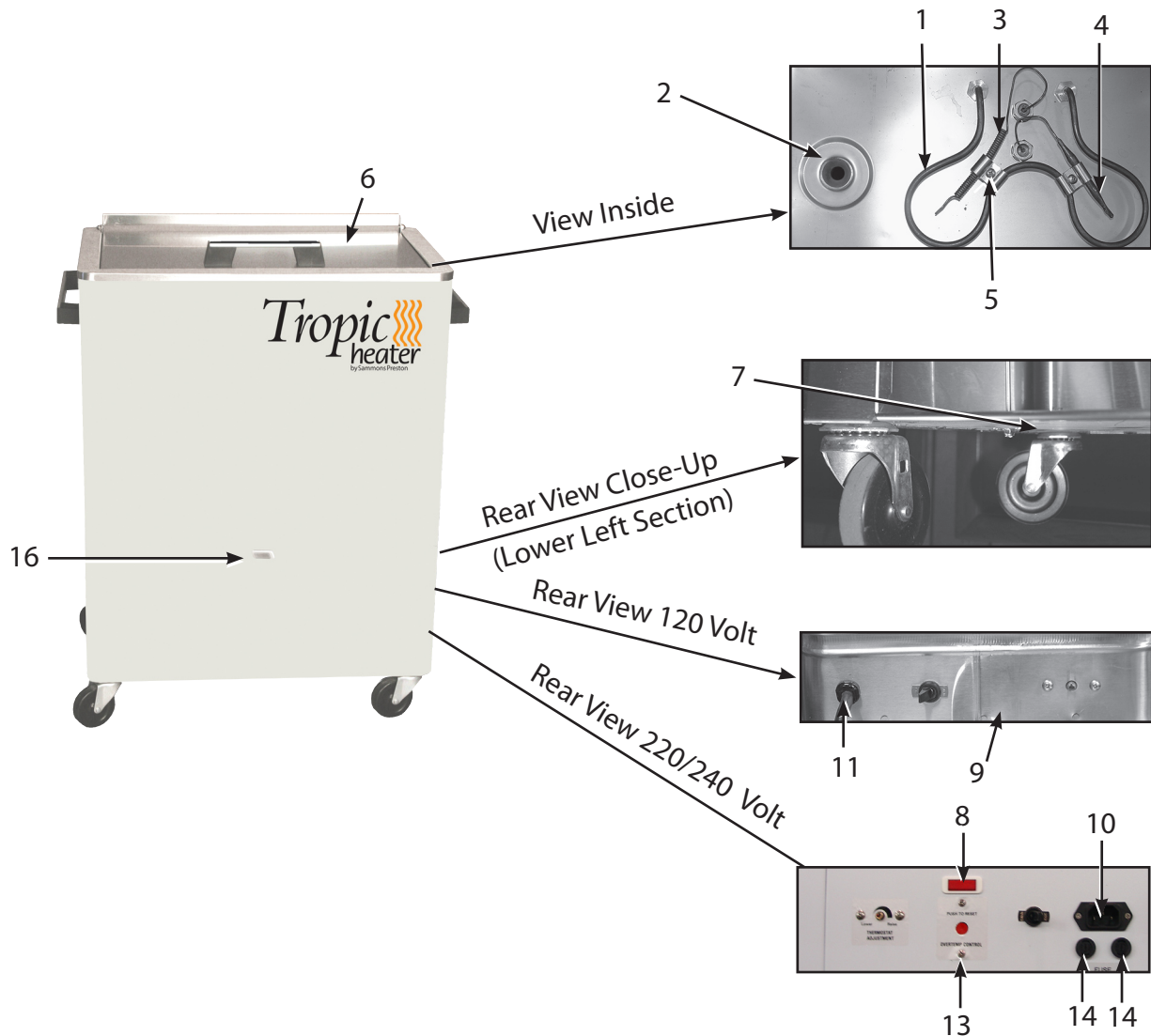

**NOTE:** Tropic Heater 12-Pac Mobile 220-240 Volt model shown; models will vary.


\* All 220-240 Volt models have this button.

# NOMENCLATURE

Tropic Heater Heating Units by Sammons Preston

## Diagram (CONTINUED)

- |                        |                                  |
|------------------------|----------------------------------|
| 17. HotPac             | 23. Side Handle                  |
| 18. Neck Pac Insert    | 24. Thermostat, Hydraulic        |
| 19. Drain, Copper Tube | 25. Thermostat Intensity Control |
| 20. Lid Hinge Nuts     | 26. On/Off Switch                |
| 21. Lid Knob/Handle    | 27. Internal Wiring Harness      |
| 22. Lid Hinge          |                                  |

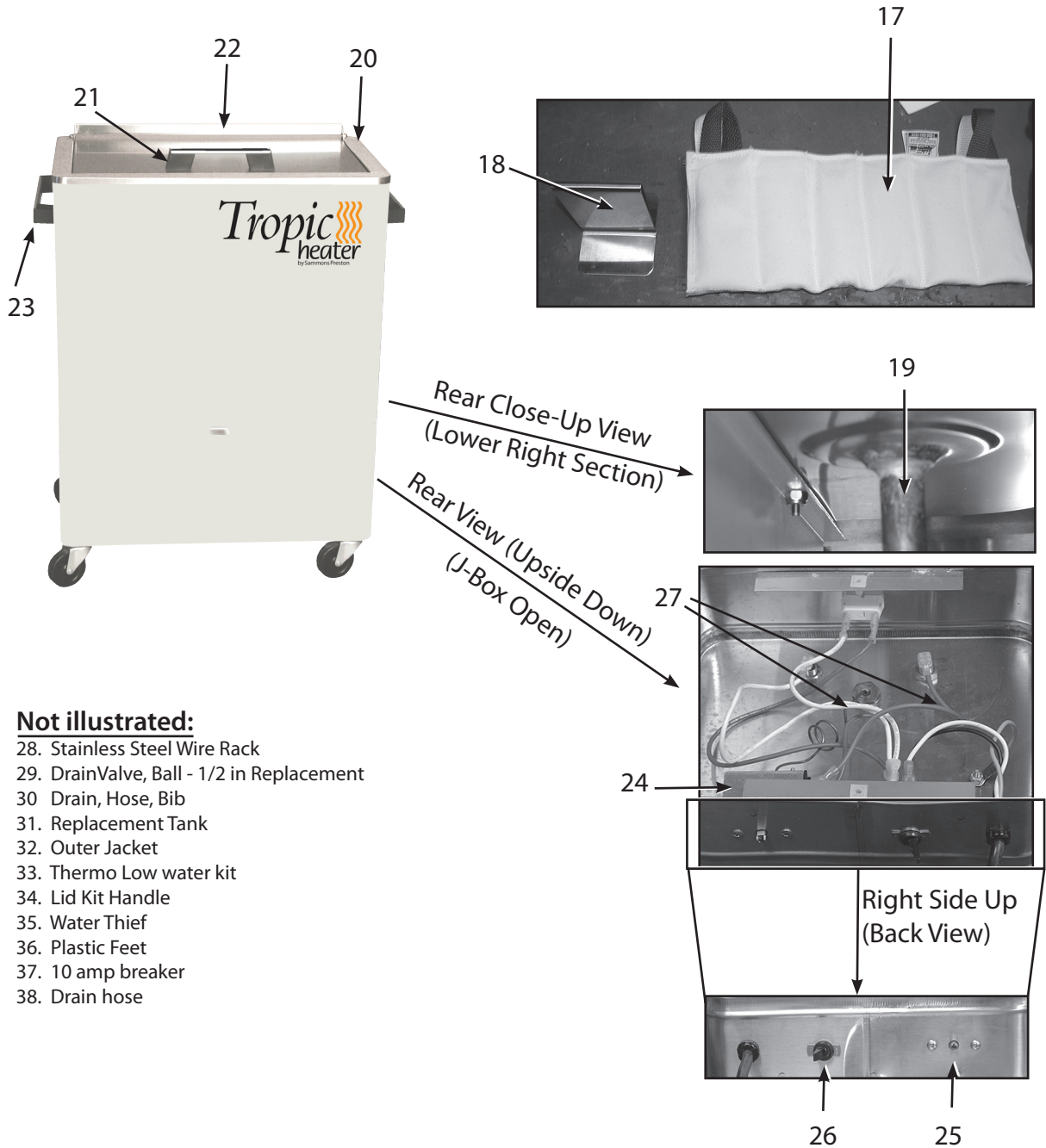**Not illustrated:**

- 28. Stainless Steel Wire Rack
- 29. Drain Valve, Ball - 1/2 in Replacement
- 30. Drain, Hose, Bib
- 31. Replacement Tank
- 32. Outer Jacket
- 33. Thermo Low water kit
- 34. Lid Kit Handle
- 35. Water Thief
- 36. Plastic Feet
- 37. 10 amp breaker
- 38. Drain hose

**3537 Tropic Heater 4-Pac**

Power Supply	110-120 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	15.44 Liters
Filled Weight	25 kg (55 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-out Temp	190° F ± 14° F (88° C ± 8° C)
Heating Up Time	Time to 160° F (70° C) - 2 Hours
Cool Down Time	Time from 160° F (70° C) - 1 Hour
Safety Class	Type B ⚡
Safety Tests	UL 544
Dimensions	16 in H x 13 in L x 8 in W (41 cm H x 33 cm L x 20 cm W)
Product Weight	14 lbs (6.4 kg)
Shipping Weight	16 lbs (7.3)

**3537V Tropic Heater 4-Pac**

Power Supply	220-240V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	15.44 Liters
Filled Weight	25 kg (55 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-out Temp	190° F ± 14° F (88° C ± 8° C)
Heating Up Time	Time to 160° F (70° C) - 2 Hours
Cool Down Time	Time from 160° F (70° C) - 1 Hour
Safety Class	Type B ⚡
Safety Tests	IEC/EN 60601-1
Dimensions	16 in H x 13 in L x 8 in W (41 cm H x 33 cm L x 20 cm W)
Product Weight	14 lbs (6.4 kg)
Shipping Weight	16 lbs (7.3)

**Standard Accessories:** Includes four Standard size Tropic Pacs.

**3540 Tropic Heater 6-Pac**

Power Supply	110-120 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	43.23 Liters
Filled Weight	60 kg (125 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-Out Temp	190° F ± 14° F (88° C ± 8° C)
Heat Up Time	Time to 160° F (70° C) - 4 Hours
Cool Down Time	Time from 160° F (70° C) - 2 Hours
Safety Class	Type B ⚡
Safety Tests	UL 544
Dimensions	20 in H x 15 in L x 13 in W (51 cm H x 38 cm L x 33 cm W)
Product Weight	31 lbs (14.1 kg)
Shipping Weight	35 lbs (15.9 kg)

**3540V Tropic Heater 6-Pac**

Power Supply	220-240V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	43.23 Liters
Filled Weight	60 kg (125 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-Out Temp	190° F ± 14° F (88° C ± 8° C)
Heat Up Time	Time to 160° F (70° C) - 4 Hours
Cool Down Time	Time from 160° F (70° C) - 2 Hours
Safety Class	Type B ⚡
Safety Tests	IEC/EN 60601-1
Dimensions	20 in H x 15 in L x 13 in W (51 cm H x 38 cm L x 33 cm W)
Product Weight	31 lbs (14.1 kg)
Shipping Weight	35 lbs (15.9 kg)

**Standard Accessories:** Includes six Standard size Tropic Pacs.

**3538 Tropic Heater 8-Pac**

Power Supply	110-120 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	31.64 Liters
Filled Weight	56 kg (120 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-out Temp	190° F ± 14° F (88° C ± 8° C)
Heating Up Time	Time to 160° F (70° C) - 3 Hours
Cool Down Time	Time from 160° F (70° C) - 2 Hours
Safety Class	Type B ⚡
Safety Tests	UL 544
Dimensions	20 in H x 18 in L x 16 in W (51 cm H x 46 cm L x 41 cm W)
Product Weight	51 lbs (23.1 kg)
Shipping Weight	54 lbs (24.5 kg)

**3538V Tropic Heater 8-Pac**

Power Supply	220-240V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	31.64 Liters
Filled Weight	56 kg (120 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-out Temp	190° F ± 14° F (88° C ± 8° C)
Heating Up Time	Time to 160° F (70° C) - 3 Hours
Cool Down Time	Time from 160° F (70° C) - 2 Hours
Safety Class	Type B ⚡
Safety Tests	IEC/EN 60601-1
Dimensions	20 in H x 18 in L x 16 in W (51 cm H x 46 cm L x 41 cm W)
Product Weight	51 lbs (23.1 kg)
Shipping Weight	54 lbs (24.5 kg)

**Standard Accessories:** Includes eight Standard size Tropic Pacs.

**5614-05 Tropic Heater 8-Pac Mobile**

Power Supply	110-120 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	49.32 Liters
Filled Weight	85 kg (140 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-out Temp	190° F ± 14° F (88° C ± 8° C)
Heating Up Time	Time to 160° F (70° C) - 4 Hours
Cool Down Time	Time from 160° F (70° C) - 2 Hours
Safety Class	Type B 
Safety Tests	UL 544
Dimensions	33 in H x 21 in L x 16 in W (85 cm H x 52 cm L x 39 cm W)
Product Weight	68 lbs (30.8 kg)
Shipping Weight	74 lbs (33.6 kg)

**Standard Accessories:** Includes eight Standard size Tropic Pacs.

**3539/ 3539X Tropic Heater 12-Pac Mobile**

Power Supply	110-120 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	68.81 Liters
Filled Weight	110 kg (190 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-Out Temp	190° F ± 14° F (88° C ± 8° C)
Heat Up Time	Time to 160° F (70° C) - 6 Hours
Cool Down Time	Time from 160° F (70° C) - 3 Hours
Safety Class	Type B ⚡
Safety Tests	UL 544
Dimensions	33 in H x 27 in L x 16 in W (85 cm H x 67 cm L x 40 cm W)
Product Weight	86 lbs (39 kg)
Shipping Weight	93 lbs (42.2 kg)

**3539V Tropic Heater 12-Pac Mobile**

Power Supply	220-240V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	68.81 Liters
Filled Weight	110 kg (190 lbs)
Temperature Range	160° F - 165° F (71° C - 74° C)
Thermal Cut-Out Temp	190° F ± 14° F (88° C ± 8° C)
Heat Up Time	Time to 160° F (70° C) - 6 Hours
Cool Down Time	Time from 160° F (70° C) - 3 Hours
Safety Class	Type B ⚡
Safety Tests	IEC/EN 60601-1
Dimensions	33 in H x 27 in L x 16 in W (85 cm H x 67 cm L x 40 cm W)
Product Weight	86 lbs (39 kg)
Shipping Weight	93 lbs (42.2 kg)

**Standard Accessories:** Includes twelve Standard size Tropic Pacs.

**NOTE:** 3539X Tropic Heater 12-Pac Mobile does not include Tropic Pacs.

## SETUP

Tropic Heater Heating Units by Sammons Preston

Remove all of the contents inside the Tropic Heater Heating Unit and then replace the wire rack. Make certain the drain valve is closed before filling the tank with water. The tank should be approximately 3/4 full with packs removed. The tank may be filled from any standard type faucet.

Check the voltage rating on the serial decal plate which is located on the back of the unit. To turn the unit on, simply plug the unit into a 120 Volt, 60 Hz or 220/240 Volt, 50-60Hz outlet as indicated on the decal plate. To turn the unit off, simply unplug from outlet. Do not attempt to use direct current. Follow the procedures indicated in the Precautionary Instructions. Do not attempt to use the unit if it is not properly grounded.

## OPERATION

The Tropic Heater Heating Unit is simple and easy to operate. The controls are located in the rear, near the bottom of the unit. The thermostat controls the temperature of the water. It is located near the bottom of the unit on the control panel. It has been tested and preset at the factory to normal operating temperature. If further adjustment is necessary, it can be easily accomplished by turning the adjusting screw located on the thermostat. ***Turning the outer screw counterclockwise lowers the temperature, while turning it clockwise raises it.***

**NOTE: Do not turn the inner stem under any circumstance! Stem is preset by the manufacturer.**

The Tropic Heater Heating Units operate through the use of a hydraulic capillary-type thermostat and can be left plugged in and turned on to provide a ready supply of heated Tropic Pacs.

The Tropic Pac rack accommodates different sizes and types of packs.

**3537/ 3537V** – The unit comes complete with four Standard Size Tropic Pacs.

**3540/ 3540V** – The unit comes complete with six Standard Size Tropic Pacs.

**3538/ 3538V** – The unit comes complete with eight Standard Size Tropic Pacs.

**5614-05** – The unit comes complete with eight standard Tropic Pacs.

**3539/ 3539V** – The unit comes complete with 12 standard Tropic Pacs.

***NOTE: 3539X Tropic Heater 12-Pac Mobile does not include Tropic Pacs.***

***NOTE: 3537/ 3537V, 3540/ 3540V, 3538/ 3538V and 5614-05 Models include rack insert that holds two Neck Contour Tropic Pacs.***

The following are solutions to minor performance problems. Only qualified personnel should service equipment problems.

<b>PROBLEM</b>	<b>POSSIBLE CAUSE</b>	<b>POSSIBLE SOLUTION</b>
Unit plugged in, water does not get hot	<ol style="list-style-type: none"> <li>1. Breaker for electric circuit off</li> <li>2. Thermostat not functioning</li> <li>3. Heating element burned out</li> </ol>	<ol style="list-style-type: none"> <li>1. Turn breaker on</li> <li>2. Replace thermostat</li> <li>3. Replace element</li> </ol>
Cloudiness of water	<ol style="list-style-type: none"> <li>1. Filler from packs is seeping</li> <li>2. Too long a period between cleanings</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace packs</li> <li>2. Drain and clean</li> </ol>
Packs too hot	<ol style="list-style-type: none"> <li>1. Thermostat set too high</li> <li>2. Thermostat failure</li> </ol>	<ol style="list-style-type: none"> <li>1. Adjust thermostat</li> <li>2. Replace thermostat</li> </ol>
Packs too cool	<ol style="list-style-type: none"> <li>1. Thermostat set too low</li> <li>2. Thermostat failure</li> </ol>	<ol style="list-style-type: none"> <li>1. Adjust thermostat</li> <li>2. Replace thermostat</li> </ol>
Unit boils	<ol style="list-style-type: none"> <li>1. Thermostat failure</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace thermostat</li> </ol>
Rust/Discoloration	<ol style="list-style-type: none"> <li>1. Contact with ferrous material</li> <li>2. Deposits on inside of tank</li> <li>3. Too much chlorine in water</li> </ol>	<ol style="list-style-type: none"> <li>1. Remove the ferrous metal item from the unit (staple, paper clip, safety pin, tongs, etc.) and clean thoroughly. Add a dechlorinating agent if local water has high chlorine content.</li> <li>2. Thoroughly clean tank and remove all deposits.</li> <li>3. Clean tank and rinse thoroughly. Add a dechlorinating agent if local water has high chlorine content.</li> </ol>

Listed below are the Power Cords for each Country, their part numbers, and their dimensions.

<b>Ref.</b>	<b>Description</b>
21284	Euro [18 AWG, 203.2 cm (80 in), Shielded]
78121	US [18 AWG, 203.2 cm (80 in), Shielded]
20971	Australian [18 AWG, 203.2 cm (80 in), Shielded]
20972	Swiss [18 AWG, 203.2 cm (80 in), Shielded]
20973	UK [18 AWG, 203.2 cm (80 in), Shielded]
20974	Danish [18 AWG, 203.2 cm (80 in), Shielded]
20975	Japanese [18 AWG, 203.2 cm (80 in), Shielded]
20976	Indian [18 AWG, 203.2 cm (80 in), Shielded]
20977	Israeli [18 AWG, 203.2 cm (80 in), Shielded]

The Tropic Heater Heating Unit is equipped with an immersion type heating element and a hydraulic capillary type thermostat which evenly maintains the Tropic Pac temperature in the water and provides a ready supply of heated packs. It is critical to maintain the water level over the top of the Tropic Pac to avoid damage to the heating element, the stainless steel, or the Tropic Pac. Water is constantly lost during operation due to evaporation. Therefore, it is essential that water be added daily. The tank should also be drained, cleaned and inspected systematically, at minimum intervals of every two weeks.

Always unplug the unit from the power source when emptying or cleaning the unit. Drain the unit by the drain valve located at the bottom rear of the unit. Either move the unit to a floor drain or attach the extension hose to it when draining. (3537/ 3537V Units have no drain and can be emptied by turning unit upside down.)

## **WARNING**

- Sammons Preston does not recommend the Tropic Heater Heating Unit be moved while filled with heated water. Tipping over could result in a burn to the user or others. If the Tropic Heater Heating Unit must be moved while filled, **exercise extreme caution!** It is recommended to allow the water to cool below 120° F (49° C) before moving the unit.
- Always unplug the Tropic Heater Heating Unit from the power source before attempting to empty water from unit.

The Tropic Heater Heating Unit is made of high quality stainless steel that, with proper care and maintenance, could easily last a lifetime. Improper care could allow the stainless steel to wear prematurely.

Follow these simple steps to ensure that the Tropic Heater Heating Unit remains in excellent condition:

1. Excessive chlorine can be detrimental to the stainless steel in the Tropic Heater Heating Unit. No bleach or any cleaner with high chlorine content should be used under any circumstance.
2. Clean regularly for optimal service and operation.  
**NOTE: Failure to maintain the unit (including regular cleaning) will void your warranty.**
3. Fill daily with water. Water is constantly lost during operation due to evaporation.
4. Chlorine in regular tap water may be present in high enough concentrations to damage the unit. If you suspect high levels of chlorine in the Tropic Heater Heating Unit, the addition of a dechlorinator is recommended.
5. Certain additives (such as herbal teas, essential oils, etc.) will damage the Tropic Pacs and components of the heating unit, including the stainless steel. The Tropic Heater Heating Unit was designed to heat water and Tropic Pacs only. The addition of any such foreign substance (except dechlorinator) to the water in the unit will void the warranty.
6. Inspect the heating element for pitting and other visible damage during regular cleaning intervals.
7. Inspect the power cord for wear, fraying, and other visible damage during regular cleaning intervals.

## To avoid potential for rusting:

1. **Do not** use bleach or cleaners with high chlorine content.
2. **Do not** use paper clips, safety pins, staples, coat hangers, etc., in, on, or near the unit.
3. **Do not** clean the unit with carbon steel wool or a carbon steel brush. Use only soft cloth, Scotch-Brite™ type scouring pads of green, blue, or white abrasiveness, or stainless steel wool if absolutely necessary.
4. **Do not** use abrasive cleaners which tend to disturb the grain in the finish, especially on the exterior of the unit. For exterior cleaning, use only non-abrasive stainless steel polish.
5. Water level is critical: Add water daily, if necessary, to keep it full.
6. Regularly clean and drain the tank (**every two weeks**). Failure to properly maintain the unit will cause premature wear and will void the warranty.
7. Remove all deposits from interior surfaces and parts. The deposits are concentrations of minerals that will allow rusting to begin. If allowed to accumulate on the heating element, these deposits will lower the efficiency of the element and increase the power consumption of the unit. However, exercise extreme care if cleaning element or thermostat(s). These components can be easily damaged without proper care or caution.
8. If you suspect that you have “hard water”, you may need to change the water and clean the unit more frequently. Salt deposits are an indication of hard water that will eventually cause rust.

## Cleaning Tips

1. The interior of the unit should be cleaned, at least every two weeks, using a low abrasive bathroom cleaner with a soft cloth or green, blue, or white Scotch-Brite™ type scouring pad. Stainless steel wool may be used if necessary, but not regular, carbon steel wool. Check for low or no chlorine content in your cleaner and make sure that the residue is thoroughly rinsed away with water.
2. A strong solution of vinegar and water may be used to dissolve away deposits, which then must be thoroughly rinsed away. Results may vary.
3. For exterior cleaning, use stainless steel polish and a soft cloth only. Numerous industrial stainless steel polishes are available from various supply companies. Use of any cleaner except stainless steel polish on the exterior of the unit is not recommended. Abrasive cleaners and scouring pads can damage the finish of the unit.

## Care of Tropic Pacs

Always return the Tropic Pac to the hot water of the heating unit after each treatment. There it is heated, kept clean, and ready for immediate use. The Tropic Pac may be boiled; however, this hastens the deterioration of the pack. The pack may also be cleaned by scrubbing the pack with soap and water. The simplified method of keeping the pack clean is to keep it immersed in water.

The Tropic Pac, under constant daily use, should have a useful life of at least one year. When the pack begins to wear out, the filler oozes and leaks through the fabric of the cloth and sections of the pack appear to be loose and empty and will not retain heat properly. The pack should be replaced when it reaches this state. Original replacement Tropic Pacs may be ordered from an authorized Sammons Preston dealer.

**NOTE:** If Tropic Pacs are to be stored for extended periods, they may be placed - while wet - in plastic bags and stored in a freezer.

# WARRANTY

Tropic Heater Heating Units by Sammons Preston

Sammons Preston ("Company") warrants that the Tropic Heater Heating Units ("Products") are free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase. If these Products fail to function during the one year warranty period due to a defect in material or workmanship, at the Company's option, the Company or the selling dealer will repair or replace this Product without charge within a period of thirty days from the date on which the Product is returned to the Company or the dealer.

All repairs to the Product must be performed by a service center authorized by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

To participate in warranty coverage, this Product's warranty registration card (included with Product) must be filled out and returned to the Company by the original owner within ten (10) business days of purchase.

### **This Warranty Does Not Cover:**

- Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service agent.
- Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service agent.
- Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and necessary maintenance or any use that is inconsistent with the Product User Manual.

### **COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To obtain service from Company or the selling dealer under this warranty:

1. A written claim must be made within the warranty period to the Company or the selling dealer.

Written claims made to the Company should be sent to:

Sammons Preston  
A Patterson Medical Company  
P.O. Box 5071  
Bolingbrook, IL 60440-5071  
1-800-323-5547  
1-800-547-4333 FAX

2. The Product must be returned to the Company or the selling dealer by the owner.

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product. Any representation or agreement not contained in the warranty shall be void and of no effect.

**THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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\* Key numbers refer to Nomenclature Section (pages 6 and 7).

\*\* Used to hold the hydraulic thermostat to the heating element.

\*\*\* Includes side handle, 10-24x3/8 screw, nylon washer, 10-32 acorn nut, 10-32x1/4 screw, and handle adapter.

## 3537V Tropic Heater 4-Pac 220/240 Volt – (Low Water)

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	21004
9	Junction Box Cover	21005
15	Pilot Lamp, Red	22147
20	Lid Hinge Rod Nuts (2 required)	21737
28	Rack, Stainless Steel	21062
21	Handle, Lid	21025
22	Hinge Rod, Lid	22668
23	Handle, Side Kit (2)	21994
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
	Pilot Lamp, Green	22146
11	Cordset (Country Specific)	See page 15
18	Insert, For Neck Contour Packs	21053
	Receptacle, Power	74723
	Carrier, Fuse	72733
	Holder, Fuse	72734
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772
	Washer, Nylon	21027

\* pages 8 and 9

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

**3540 Tropic Heater 6-Pac 120 Volt**

Key*	Part Description	Part No.
1	Lid	23457
3	Pilot Lamp, Clear	22148
4	Power Cord	20568
5	On/Off Switch	23406
7	Drain Valve	21373
8	Element, Heating 120 Volt, 1000 Watt	10630
10	Hydraulic Thermostat	10631
11	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
12	Internal Wiring Harness	21031
13	Drain, Copper Tube Part A	21288
14	Handle, Lid Kit	20512
20	Junction Box (J-Box) Cover	23460
21	Stainless Steel Wire Rack	22259
25	Water Thief	21882
27	Hose, Drain	21635

\* Key numbers refer to Nomenclature Section (pages 6 and 7).

\*\* Used to hold the hydraulic thermostat to the heating element.

\*\*\* Includes decorative handle and 1/4-20x1-1/4 screw.

**3540V Tropic Heater 6-Pac 220/240 Volt – (Low Water)**

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	23457
9	Junction Box Cover	23460
10	Cordset (Country Specific)	See page 15
38	Hose, Drain	21635
19	Drain, Copper Tube Part A	21288
21	E-2 Lid Handle	20512
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
29	Drain Valve, Ball - 1/2 in Replacement	21373
35	Water Thief (2)	21882
26	On/Off Switch	23406
	Lamp, Green, Lens	22146
15	Lamp, Red, Lens	22147
	Carrier, Fuse (2)	72733
	Holder, Fuse (2)	72734
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772
	Lid, Rod	90796
18	Insert, For Neck Contour Packs	21053
28	Rack, Wire	22259

\* pages 8 and 9

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## 3538 Tropic Heater 8-Pac 120 Volt

Key*	Part Description	Part No.
1	Lid	22219
2	Lid Rod	22231
3	Pilot Lamp, Clear	22148
4	Power Cord	20568
5	On/Off Switch	23406
7	Drain Valve	21373
8	Element, Heating 120 Volt, 1000 Watt	10630
10	Hydraulic Thermostat	10631
11	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
12	Internal Wiring Harness	21031
13	Drain, Copper Tube Part A	21288
14	Handle, Lid Kit***	22171
15	Lid Rod Nuts (2 required)	21737
20	Junction Box (J-Box) Cover	22223
21	Stainless Steel Wire Rack	22232
25	Water Thief	21882
26	Feet, Plastic	75538
27	Hose, Drain	21635
28	Drain, Copper, Part B	21289

\* Key numbers refer to Nomenclature Section (pages 6 and 7).

\*\* Used to hold the hydraulic thermostat to the heating element.

\*\*\* Includes decorative handle, nut 10-32 acorn brass, screw 10-32x2 and flat plastic washer.

## 3538V Tropic Heater 8-Pac 220/240 Volt – (Low Water)

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
9	Junction Box Cover	22223
38	Hose, Drain	21635
15	Pilot Lamp, Red	22147
19	Drain, Copper Tube Part A	21288
20	Lid Hinge Rod Nuts (2 required)	21737
22	Lid Rod	22231
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
29	Valve, Boiler Drain, 1/2 I.D. Chrome	21373
	Carrier, Fuse	72733
	Holder, Fuse	72734
34	Handle, Lid finished	20531
35	Water Thief	21882
26	On/Off Switch	23406
	Lamp, Green	22146
	Cordset (Country Specific)	See page 15
28	Rack, Wire	22232
	Drain, Copper, Part B	21289
18	Insert, For Neck Contour Packs	21053
	Tank, Replacement, w/Drain	23385
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772

\* pages 8 and 9

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

**5614-05 Tropic Heater 8-Pac Mobile 120 Volt**

Key*	Part Description	Part No.
1	Lid	22219
2	Lid Rod	22231
3	Pilot Lamp, Clear	22148
4	Power Cord	20568
5	On/Off Switch	23406
7	DrainValve, Ball - 1/2 in Replacement	21085
8	Element, Heating 120 Volt, 1000 Watt	10630
10	Hydraulic Thermostat	10631
11	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
12	Internal Wiring Harness	61579
13	Drain, Copper Tube Part A	21286
14	Handle, Lid Kit***	22171
15	Lid Rod Nuts (2 required)	21737
16	Handle, Side (2)	20237
17	Caster, 3-inch	21259
20	Junction Box (J-Box) Cover	22087
21	Stainless Steel Wire Rack	22259
22	Drain, Hose, Bib	23392
23	Replacement Tank	23385
24	Outer Jacket	22425
25	Water Thief	21882
27	Hose, Drain	21635

\* Key numbers refer to Nomenclature Section (pages 6 and 7).

\*\* Used to hold the hydraulic thermostat to the heating element.

\*\*\* Includes decorative handle, nut 10-32 acorn brass, screw 10-32x2 and flat plastic washer.

**3539/ 3539X Tropic Heater 12-Pac Mobile 120 Volt**

Key*	Part Description	Part No.
1	Lid	22297
2	Lid Rod	22303
3	Pilot Lamp, Clear	22148
4	Power Cord	20568
5	On/Off Switch	23406
7	Drain Valve, Ball - 1/2 in Replacement	21085
8	Element, Heating 120 Volt, 1000 Watt	10630
10	Hydraulic Thermostat	10631
11	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
12	Internal Wiring Harness	61579
13	Drain, Copper Tube Part A	21286
14	Handle, Lid Kit***	22171
15	Lid Rod Nuts (2 required)	21737
16	Handle, Side	20237
17	Caster, 3-inch	21259
20	Junction Box (J-Box) Cover	22087
21	Stainless Steel Wire Rack	22304
22	Drain, Hose, Bib	23392
23	Replacement Tank	56314
24	Outer Jacket	22421
25	Water Thief	21882
27	Hose, Drain	21635

\* Key numbers refer to Nomenclature Section (pages 6 and 7)

\*\* Used to hold the hydraulic thermostat to the heating element.

\*\*\* Includes decorative handle, nut 10-32 acorn brass, screw 10-32x2 and flat plastic washer.

## 3539V Tropic Heater 12-Pac Mobile 220-240 Volt – (Low Water)

Key*	Part Description	Part No.
1	Element, Heating, 220 Volt~, 1000 Watt	22245
4	Thermostat, Hydraulic	10631
3	Lamp, Pilot, Green	22146
11	Cord Set (Country Specific)	See page 15
26	Switch	23406
28	Rack	22304
	Hose, Nylon 5/8 x 4' Coupled M-F	21635
7	Caster, 3 inch	21259
21	Handle, M/SS, Lid, Finished	20513
23	Handle, Side, Common (2)	20237
34	Lid, Assembly With Handle	22297
9	Cover, Junction Box, M-2, 220-240 Volt	22087
19	Drain, Copper M-2 Part A #25010	21286
5	Bracket, Bulb Thermostat, S/A (Lo-Water) (2)**	21023
35	Water Thief	21882
	Spacer, Thermostat (Lo-Water) (2)	72393
	Thermostat, (Lo-Water)***	20548
15	Lamp, Red, Indicator (Lo-Water)	22147
14	Fuse, 6.3 Amp. (2)	71772
	Fuse Holder (2)	72734
	Receptacle, Power 8843-FSG-W-40-60	74723
27	Harness Wiring	61579
31	Tank, Replacement with Drain	23373
18	Insert, For neck Contour Packs	21053
20	Lid, Rod Nuts (2)	21737
22	Lid, Rod	22303
29	Valve, Ball	21085
	Fit Bib, SS-2, M-2, M-4	23392

\* pages 8 and 9

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

Sammons Preston  
A Patterson Medical Company  
P.O. Box 5071  
Bolingbrook, IL 60440-5071  
1-800-323-5547  
1-800-547-4333 FAX

18202C