Rev. 2 - 02/11_100504

Bauerfeind AG
 Weissendorfer Strasse 5
 07937 Zeulenroda-Triebes
 Germany
 Phone +49 (0) 36628-66-40 00
 Fax +49 (0) 36628-66-44 99
 E-mail info@bauerfeind.com

MalleoTrain®

Aktivbandage
 Active support
 Orthèse active
 Vendaje activo

Material
Silikon*
Polyamid (PA)
Elastodien (ED)
Viskose (CV)
Baumwolle (CO)
Elastan (EL)
Polyurethan (PUR)
Polyester (PES)
PES + Kleber Griltex 1A**
Druckfolie

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EN Dear valued customer,

Thank you for choosing a product from Bauerfeind. MalleoTrain® is a high-quality medical product.

Please carefully read through the following instructions for use. Should you have any questions please contact your doctor, medical retailer or our Customer Service department.

MalleoTrain®, an active ankle support, produces its special effect primarily during movement and therapeutic exercises. The anatomically contoured viscoelastic inserts (pressure pads) compress and massage the soft tissues of the joint. Enhanced circulation in the joint and promotion of lymph drainage lead to reduced swelling, pain relief and improved function.

⚠ MalleoTrain® must be fitted for initial use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Post-operative and post-traumatic irritation (e. g. after sprains)
- Joint effusion and swelling associated with degenerative changes in the ankle (osteoarthritis) and joint inflammation (arthritis)
- Muscle and tendon disorders (tendomyopathies)
- Ligamental weaknesses

Side effects

With the assumption of proper fitting, no side effects involving the body as a whole have been reported to date. Supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions, harmful to health have not been reported to date. For patients experiencing the following conditions, such supports and orthoses should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the legs/feet, "e. g., Diabetes".
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin, located remotely from the fitted aid.

Fitting MalleoTrain®

1. Fold the upper part of the support outwards until you can grip the heel.
2. Pull the support over the foot up to the heel.
3. Fold back the folded-over part of the support.
4. Then pull it up until the bones of the ankle rest in the rounded hollows of the pads.
5. Adjust the fit of the support as necessary.

⚠ Important Information

- MalleoTrain® is a prescribed product that should be applied under a physician's guidance. MalleoTrain® should only be applied in accordance with these instructions for use and for the specified areas of application.
- The product is intended to be fitted only once, to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non compliance can reduce the performance of the product, thus no liability will be accepted.
- Should you notice any unusual changes (e. g., increased symptoms), please contact your medical provider without delay.
- A combination with other products (e. g., compression stockings) must be discussed beforehand with your medical provider.
- The product's unique benefits are demonstrated primarily during physical activity. The product should be removed during prolonged periods of rest (e. g., sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances that include grease or acids.
- The product can be disposed of without any restrictions.

Product Care Instructions

⚠ All Train products² can be machine washed separately at 30°C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing.

We recommend using a mesh bag (spin cycle: maximum of 500 revolutions per minute). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been tested by our in-house quality management system. However, should you have any complaint, please contact our local expert adviser or our customer service department.

Version: February 2011

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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