



Wrist Brace

Application Instructions

1. Measure the wrist circumference to determine the proper size. Slide brace on the affected hand with the layer of stockinette on the back of the hand.
2. Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.
3. Trim the straps leaving approximately 1" - 2" of overlap.

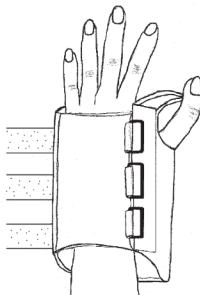

NOTE: The metal stay has been positioned so the wrist will be set at 30° of extension. For patients with Carpal Tunnel Syndrome, the metal stay can be adjusted to 0° to 20° of extension.

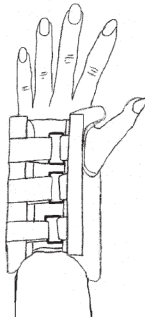

WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.