

*Wenzel<sup>Y</sup>te Re/hab*

drive  
WWW.DRIVEMEDICAL.COM

basket for wenzelite  
safety rollers

item # CE 1315



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

*Wenzel<sup>Y</sup>te Re/hab*

drive  
WWW.DRIVEMEDICAL.COM

---

## installation

---

1. Remove bolts from clips.
2. Mount basket so that rim sits on the top front of handlebar. Install clips over the basket making sure that the rim of the basket sits firmly in dedicated spot on clip.
3. Return bolts with the flat part facing inside of walker and nut facing the inside of basket.



## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**drive**  
WWW.DRIVEMEDICAL.COM

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China