

# lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

O<sub>2</sub> attachment  
item #CE 1250

for use with  
wenzelite safety rollers



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

*Wenzelite Re/hab*

[www.drivemedical.com](http://www.drivemedical.com)

drive  
MEDICAL DESIGN & MANUFACTURING

## installation instructions

Install both brackets "A" on walker as per diagram. The larger section of the bracket mounts onto the safety roller.



Mount O2 attachment into smaller section of the brackets, making sure it rests on frame of safety roller. Tighten all screws.



## inserting O2 tank

Loosen knobs "B" and insert tank. Tighten knobs.

