

postpartum support

Loving Comfort

To Wear

The loving Comfort® Postpartum Support is designed so that it is easy to put on with out assistance. Grip your loving Comfort® Postpartum Support at each end and place the "loop" pad over your abdomen as shown. Stretch, and touch the "hook" to the "loop" and your support is secure. You may vary the tension for maximum comfort by adjusting the amount of stretch.



Size	Waist(in.)
S	24"-30"
M	32"-38"
L	40"-46"
XL	48"-54"

Washing instructions

Hand wash in cold or warm water with mild detergent. Hang dry. Take care to cover the Velcro hook so it doesn't tear the fabric as you handle it. Do not machine wash or dry.

Always consult your physician when ever you have questions about your health or your baby's health.