# AIRCAST® AIRSELECT™ SHORT

ENGLISH .....	2
ESPAÑOL .....	4
DEUTSCH.....	6
ITALIANO.....	8
FRANÇAIS .....	10
NEDERLANDS.....	12
SLOVENČINA .....	14
DANSK .....	16
SVENSKA.....	18
SUOMI .....	20
ČEŠTINA .....	22
PORTUGUÊS.....	24
NORSK.....	26
简体中文.....	28
日本語 .....	30

**AIRCAST®**

DJO, LLC  
A DJO Global Company  
1430 Decision Street  
Vista, CA 92081-8553 • USA

T 760.727.1280 • F 760.734.5608  
DJOglobal.com

©2012 DJO, LLC

01E950\_B

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USE / INDICATIONS:

The SHORT walking brace is effective for the treatment of forefoot, midfoot, and hindfoot injuries; foot fractures; soft tissue, tendon or ligament injuries or procedures; acute or post-operative care (ie. bunionectomy, removal of a bunion, osteotomy, removal of bone near a damaged joint, etc.); and ankle sprains. Effective in fracture healing and the management of edema (swelling).

### WARNINGS AND PRECAUTIONS:

Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy (a diabetic nerve condition). Check your skin frequently for "hot spots" and skin irritation. If you see any skin changes or feelings of discomfort, reduce the aircell compression. Do not use this device if you cannot feel sensations whether due to post-op anesthesia, nerve, skin or other conditions.

Use caution when walking on slippery or wet surfaces to avoid injury.

### APPLICATION INFORMATION:

*A cotton or cast sock will enhance comfort – NOT INCLUDED*

#### 1) PREPARE BRACE –

- Unfasten straps, remove front panel, and open liner.

#### 2) APPLY BRACE –

- While seated, place foot in brace. Position heel in back of brace. Wrap liner around foot and then ankle (Fig. A).
- *Replace front panel:* Position front panel so the Aircast logo is over the toes and the foot panel is situated inside the brace. The upper part of the panel should be secured to the liner (Fig. B).
- Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.

#### 3) ADJUST AIRCELL COMPRESSION –

*The SHORT walking brace aircells are initially deflated*






- Begin inflation on injured side first.
- To inflate aircell, press and release **large** inflation pump found on either side of brace (Fig. D). Pump until aircell feels snug.
- Repeat on other side. Pump until aircell feels snug.
- In the event of over-inflation, deflate aircells.

#### 4) DEFLATE AIRCELLS –

- To deflate aircells, press and hold finger on **small** release button (Fig. E).
- Deflate until pressure is comfortable.

NOTE: Deflate aircells **before removing brace** for proper fit when reapplying.

NOTE: Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

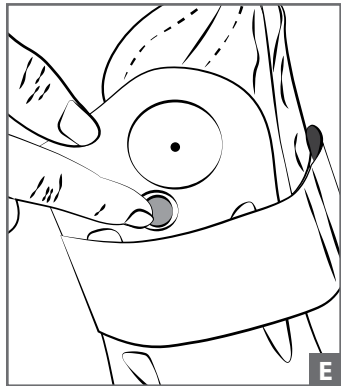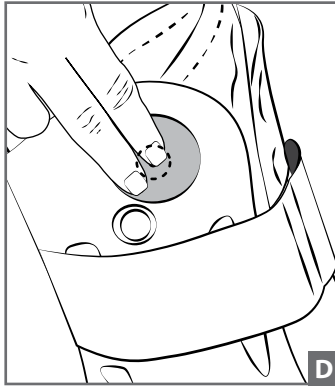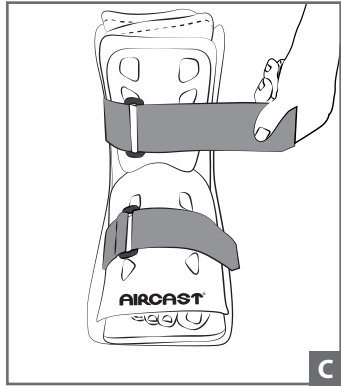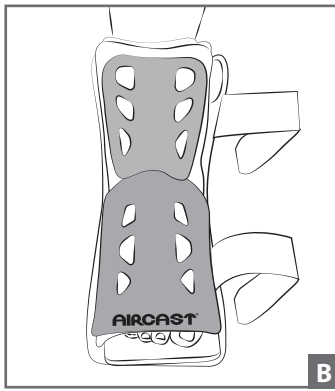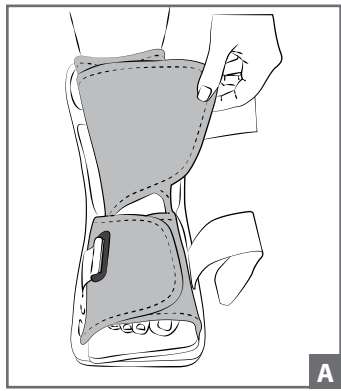### **CLEANING INSTRUCTIONS:**

THE FOAM LINER MAY BE HAND-WASHED IN 86°F/30°C WATER WITH MILD DETERGENT AND AIR-DRIED. DO NOT PUT IN DRYER OR USE OTHER HEAT SOURCE TO DRY.

### **HIGH ALTITUDE:**

AT HIGH ALTITUDES THE AIRCELLS WILL EXPAND BEYOND THEIR OPTIMAL LEVEL. ADJUST AIRCELL PRESSURE BY TIGHTENING OR LOOSENING THE STRAPS.

### **WARRANTY:**

DJO Global, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

### **LATEX FREE**

### **INTENDED FOR USE ON A SINGLE PATIENT**

### **MATERIAL COMPONENTS**

Shell: POLYPROPYLENE

Front Panel: HIGH DENSITY POLYETHYLENE

Straps: NYLON

Inflation Pumps: THERMOPLASTIC POLYURETHANE

Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)

Binding: POLYESTER

Outer Sole / Insole: EVA/RUBBER

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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