Instructions for use

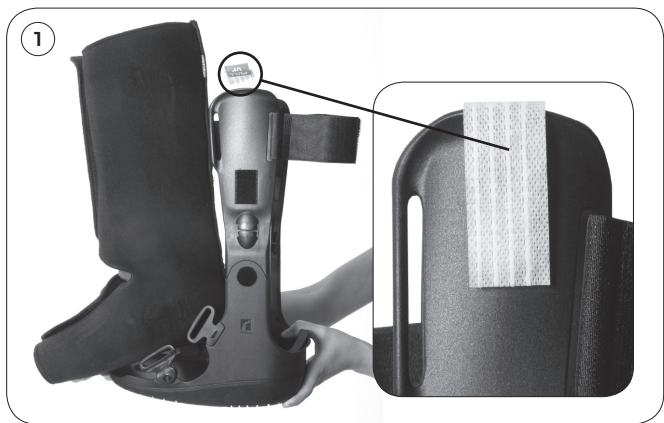
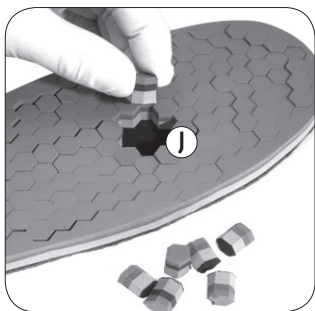
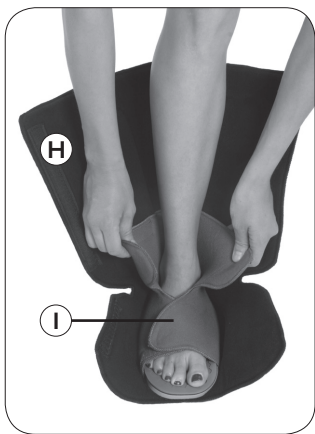
DH Active
Offloading Walker®

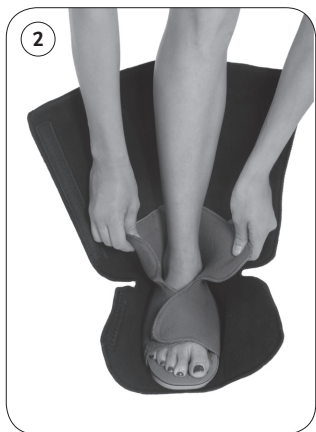
Life Without Limitations®

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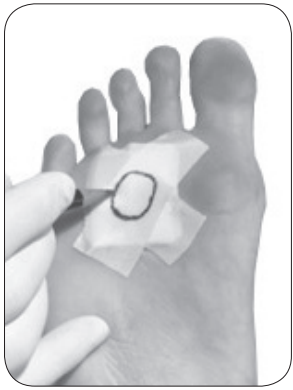
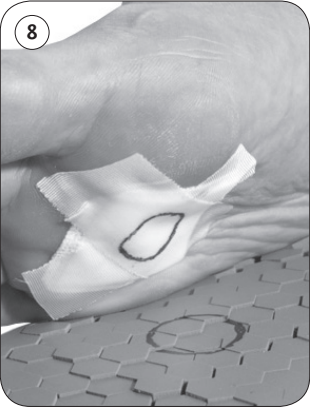
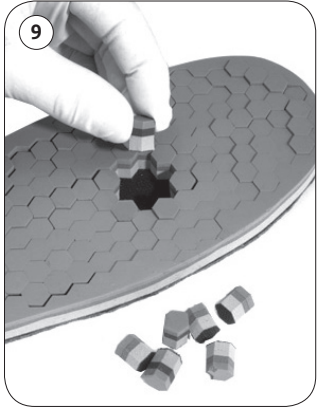
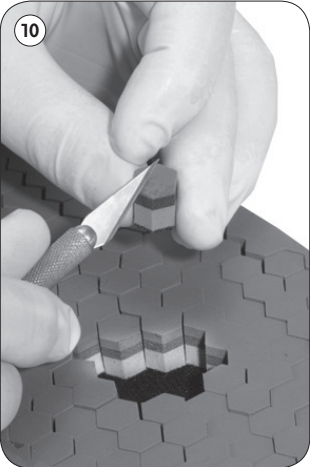
ENGLISH

INDICATIONS

Treatment of plantar ulcers, diabetes, arthritis, wound prevention, neuropathy, other conditions recommended by a physician

CAUTION

- This product should only be used when prescribed by a physician.
- Hexplugs should only be removed or altered when instructed by the prescribing physician.
- Modifications to this product are to be made only under the prescribing physician orders.
- All patients should be monitored closely.
- Patients should be instructed to inspect their feet and this product before and after each use.
- This product was designed for plantar ulcer healing.
- Wounds located at the beginning or the end of the gait path should be monitored closely to ensure that pressure relief is maintained.
- If a patient displays poor compliance, the use of this product should be reconsidered.
- Do not relieve too large of an area that may allow the ulcer to bottom out while weight bearing.
- The DH Offloading insole should not be used in conjunction with any other walker or footwear device.
- In order to prevent individual Hexplugs from falling into the relieved area, the corners of the Hexplugs lying on the perimeter of the relieved area may be trimmed (scythed) upon the prescribing physician's orders.
- Do not remove the foam pads that are adhered to the Walker's frame.
- If skin irritation of any kind develops or any other problem occurs with the use of this product, please immediately contact your medical professional.
- Do not remove walker unless instructed by your physician.
- Be sure to readjust the walker straps as necessary so that the Equalizer® feels stable and comfortable.
- This product has been designed and tested for single patient usage and is not recommended for multiple patient usage.

PHOTO CAPTIONS

- A) Walker Frame
- B) Contact Closure Leg Strap #1
- C) Contact Closure Leg Strap #2
- D) Contact Closure Leg Strap #3
- E) Contact Closure Leg Strap #4
- F) Contact Closure Leg Strap #5
- G) Vertical Strut
- H) Softgood with Preinflated Air Bladder
- I) Inner Bootie
- J) DH Offloading Insole, with Hexplugs
- K) Strut Release Strips. Do not remove until step 6

FITTING INSTRUCTIONS

1. Release contact closure straps #1, #2 and #5. Remove contact closure straps #3 and #4. Remove softgood. Do not remove the strut release strips at this time.
2. Place lower leg and foot inside softgood. Close the inner bootie snugly using the contact closures.
3. Close softgood snugly over the inner bootie using the contact closures on the softgood.
4. Insert the foot into the walker frame by gently spreading the vertical struts. Do not remove the strut release strips at this time.
5. Be sure the vertical struts are centered over the malleoli (ankle bone).

- Be sure the ankle is centered between the vertical struts.
6. Remove the strut release strips by pulling upward. This allows the softgood to be secured into position.
 7. Remove foot from walker. Cover wound with material such as a transparent film dressing, tape, gauze, etc. Mark material with a transferable mark using iodine, lipstick, indelible ink pencil, etc.
 8. Place foot inside walker. Ensure proper location of patient's foot before transferring mark to Insole's surface. Fasten walker softgood. Bear weight to ensure transfer of relief zone mark. After patient returns to a seated position, remove walker from foot.
 9. Remove marked Hexplugs from insole. **CAUTION:** *this product should not be used, if the relieved area is too large and inhibits pressure relief by allowing the ulcer or wound to bottom out.*
 10. Scythe the edges of Hexplugs surrounding the relief zone with a cutting tool such as a scalpel or scissors. **TIP:** to provide additional support, a transparent film dressing can be placed over the relief zone, or, a low-density foam can be used as filler inside the relief zone, no more than ½" (1cm) thick.
 11. Reinsert foot into walker and secure the contact closures on the inner bootie and softgood. Fasten straps in the following order: #1, #2, #3, #4, #5. **CAUTION:** *Do not pull straps too tightly as they can cause pain discomfort and /or lack of circulation.*

WASHING INSTRUCTIONS

- Remove the entire insole from the walker. **DO NOT REMOVE ANY ADDITIONAL HEXPLUGS.**
- Rinse the insole with 86F (30°C) warm water. **DO NOT USE A WASHING MACHINE. DO NOT USE ALCOHOL OR SOLVENTS TO CLEAN.**
- The insole should be air dried. To speed up the drying process, a hair dryer may be used at the lowest temperature setting. **DO NOT REMOVE ANY ADDITIONAL HEXPLUGS WHEN DRYING. DO NOT USE A DRYING MACHINE.**
- Once the insole is fully dried it can be reinserted into the walker. Follow step 11, fitting instructions, to continue use. The appropriate instructions for use should always be followed when reinserting the DH Offloading Insole.
- To clean the Equalizer® walker frame, use a damp cloth and mild soap and 86F (30°C) warm water. Hand or air dry only.
- When prescribed, the softgood may be removed for hand washing, with 86F (30°C) warm water and mild soap. Air dry only.

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Össur Iberia S.A.