



with
PARASILK
paraffin

WE ARE EXCITED TO INTRODUCE OUR FIRST MADE IN THE USA PARASILK® PARAFFIN WAX TREATMENTS.

ABOUT GLOVE TREAT® WITH PARASILK® PARAFFIN PRODUCTS

gLOVE Treat® with Parasilk® paraffin are products of Paraffin International Inc., the company that is redefining the future of paraffin wax treatments with Parasilk® paraffin products and delivery systems for spa and medical professionals. We have three unique delivery systems that eliminate the need for a paraffin pot.

3 UNIQUE DELIVERY SYSTEMS - NO PARAFFIN POT NECESSARY

GLOVE



MITT



BOOT



- Made with our trade secret Parasilk® ingredients, a blend of Medical Grade Paraffin Waxes, Virgin Coconut oil and essential oils.
- New Zipper system in gloves, boots and mitts allows for an even more convenient multiple use treatment when heating.
- Soft plush inner liner eliminates mess and keeps warmth in for up to 10 minutes while giving a therapeutic and deep moisturizing treatment.
- Our hook and loop system closes around your ankle and wrist to keep your warm Parasilk® paraffin wax enclosed around your hands and feet.
- Our Thermo Indicator in a heart shape lets you know when the product is safe to use.

THERAPEUTIC BENEFITS OF PARAFFIN WAX TREATMENTS

- Brings warmth to help increase blood flow and relax stiff joints and sore muscles.
- Increases circulation which helps flush toxins. Great for post sports recovery.
- Hydrating and skin softening elements aid in dry cuticles callused heels and cracked skin, giving hands and feet a more youthful appearance.

FOR QUESTIONS EMAIL SERVICE@GLOVETREAT.COM OR CALL 800-414-7115



PARASILK

PFN# 10304 Patent No. US 8,613,732 B2



PROFESSIONAL TREATMENT INSTRUCTIONS



HOT TOWEL CABINET WARMING DIRECTIONS

- Place two gLOVE Treat® products in your hot towel cabinet on a towel 30 minutes prior to use.
- Remove gLOVE Treat® product 2 minutes prior to use.
- Set aside until thermochromatic indicator is dark and the Parasilk® fill is a creamy or milky color but still warm to the touch.

MICROWAVE WARMING DIRECTIONS

- Place one gLOVE Treat® product in the microwave for 2 minutes. Flip after 1 minute. NOTE: Ensure plush fastener is removed prior to heating.
- Product is completely heated when Parasilk® fill is clear and indicator is red. Rub the outside of the bag gently to mix if not all of the fill is clear.
- Set aside until thermochromatic indicator is dark and the fill is a creamy or milky color but still warm



PERFORMING THE TREATMENT

- Place paper towel and towel under each hand/foot.
- Place the gLOVE Treat® product over the hand/foot and wrap with a towel to treat for 10 minutes.



REMOVING THE TREATMENT

- Carefully remove the gLOVE Treat® product and set to the side.
- Use the remaining oil to massage the area.
- Pat dry to remove any excess oil.
- Dispose of the gLOVE Treat® product into the waste receptacle.



STORING INSTRUCTIONS

THIS PRODUCT HAS A SHELF LIFE OF TWO YEARS.

gLOVE Treat® products should be stored in a cool dark place at room temperature.



PARASILK