Rolyan® Preformed Posterior Elbow Splint: A517-4 through A517-7

INDICATIONS

- For static elbow positioning

CONTRAINDICATIONS

- Not in the presence of fixed contractures
- Not for immediate treatment following fractures

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the straps are not so tight that they interfere with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. The straps can be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.


PROCEDURE

Apply splint with shorter portion on upper arm.

Modification

To trim or modify the splint, which is made from 1/8" (3.2mm) Polyform® splinting material, spot-heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for one minute or until pliable.

Heating the entire splint may cause it to lose its preformed shape.

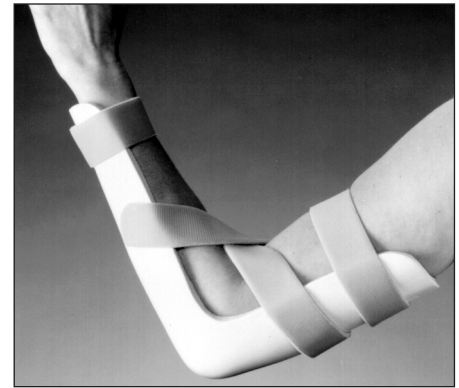A

Strapping

- Attach self-adhesive straps distally and proximally.
- For center strap, attach one hook tab to posterior end of humeral splint portion, and attach two hook tabs to sides of forearm portion. Attach strap to posterior hook tab and cross it over inside of elbow.
- Suggested strapping is shown in A.

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Patterson Medical, Ltd.