## Rolyan® PIP Ligament Repair Splint:

A602-1 through A602-4

### INDICATIONS

- To limit radial and ulnar deviation of the PIP joint while allowing full PIP flexion and extension

### CONTRAINDICATIONS

- Not in the presence of unstable fractures

### PRECAUTIONS


- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**

### CARE AND CLEANING

The splint can be cleaned with soap and lukewarm water. The straps can be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

### APPLICATION

1. To make any necessary adjustments in width of splint, bend by hand or with needle nose pliers. Make sure axis of joints on both sides of splint are aligned.
2. Apply splint to patient, lining up hinge with axis of PIP joint.
3. Apply strap with metal ring distally and strap without ring proximally.
4. A properly applied Rolyan PIP Ligament Repair Splint is shown in **A**.

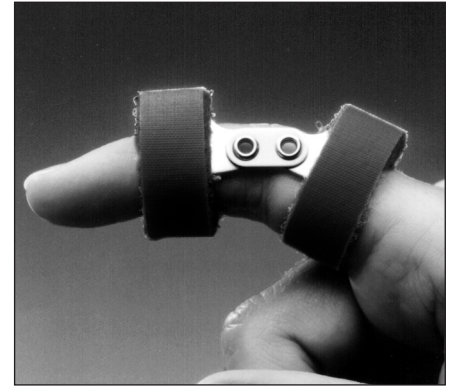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**Patterson Medical, Ltd.**