



Neoprene Products

Precautions

- Benik's products **DO NOT** contain latex.
- Allergic reactions generally appear within the first few hours. Reactions that occur days or weeks later can often be attributed to heat, moisture or hygiene issues.
- If unusual swelling, skin discoloration, rash or discomfort occur, discontinue use and consult a healthcare professional.
- Care should be taken to ensure that the support does not fit so tight that circulation is impaired.
- Not for people who have an allergic reaction to neoprene, or who have a tendency to develop dermatitis.
- Not for use over open wounds and infected areas.

Care and Cleaning

- Wash frequently to remove body oils and other soil.
- Hand wash in warm water and mild soap. Rinse thoroughly and air dry. Do not wring. Be sure support is completely dry before reapplying it.
- Do not use ointments or oils under the support.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

*Contact the Benik customer service department at **1-800-442-8910** with any questions, comments or concerns.*

11871 Silverdale Way NW, #107 Silverdale, Washington 98383 U.S.A.
Phone (360) 692-5601 • Fax (360) 692-5600
www.benik.com • info@benik.com