
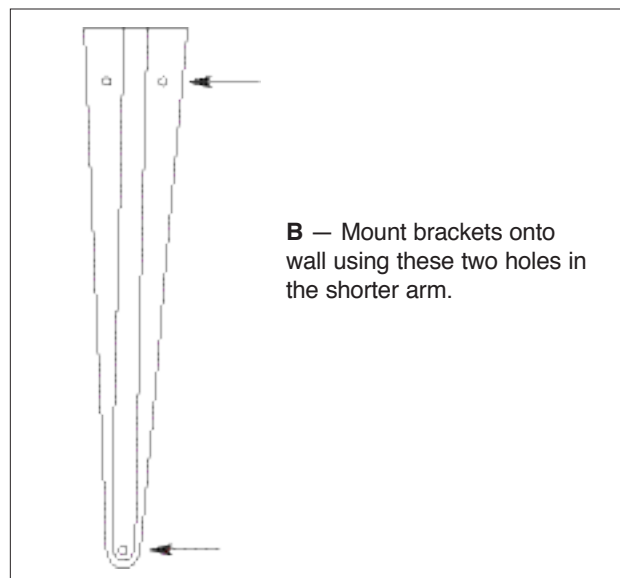


Wall Brackets for Mounting Rolyan® Strapping Carousel:

A290-2

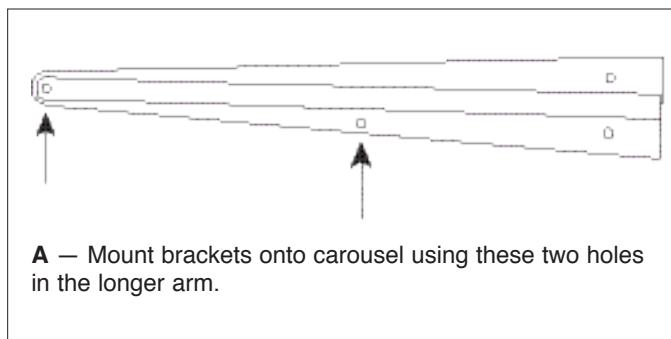

The Rolyan Strapping Carousel can sit on a table or countertop. These wall brackets should be used if wall mounting is preferred.

The brackets must be attached to support beams (studs) in the walls to adequately support the unit.

PROCEDURE

The wall brackets include the following:

- 2 brackets
- 4 short screws with nuts
- 4 long screws

Additional items required:

- electric drill with 1/8" drill bit
- Phillips screwdriver
- adjustable wrench

1. Remove cover, partitions, and strapping materials from the carousel.
2. The longer (13" [33cm]) arm of the brackets should be attached to the bottom of the carousel, and the shorter (11" [28cm]) arm should be mounted onto the wall.
3. Attach the longer arm of the brackets to the carousel using the short screws and nuts as follows:
 - Line up the two holes in the brackets that are indicated in **A** with the pre-drilled holes in the bottom of the carousel.
 - With the screw heads inside the carousel and the nuts outside the brackets, mount the brackets securely to the carousel.
4. Locate the studs in the wall. Because of the positioning of the holes already used in the bottom of the carousel, the wall brackets will be aligned on 16" (40.6cm) centers. (If the distance between the studs is more or less than 16" (40.6cm), an adapter plate will have to be installed before mounting the carousel—consult the maintenance people for the building for help.)
5. Position the carousel against the wall, aligning the brackets with the studs. Check to be sure the carousel is level. Mark two holes for each bracket (as indicated in **B**) on the wall.
6. Drill holes as marked.
7. Use the long screws to mount the brackets to the wall.



Distributed by:
Patterson Medical
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 855-393-7600

Manufactured for:
Patterson Medical
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative
Patterson Medical, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784