
Rolyan® Preformed Resting Pan Mitt Splints

Rolyan® Deluxe Preformed Resting Pan Mitt Splint: A764-1 through 6 and A764-19 through 24

Rolyan® Aquaplast® Resting Pan Mitt Splint: A763-001 through 006

Rolyan® AirThru® Resting Pan Mitt Splint: A763-010 through 015

Rolyan® Preformed Resting Pan Mitt Splint: A763-1 through 6

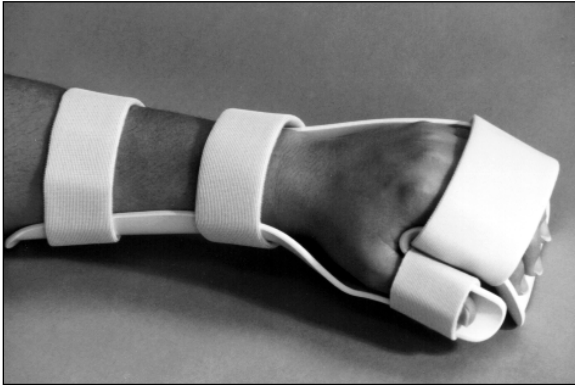A

B

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.

INDICATIONS

- To help prevent deformity—possible indications include arthritis, hypertonicity, peripheral nerve injuries, and acute synovitis

CONTRAINDICATIONS

- Not for the acute phase of most wrist or hand fractures

PRECAUTIONS

- These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the straps are not so tight that they interfere with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. The straps can be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

PROCEDURE

Fit

These splints can be modified for a better fit. Check the following areas for appropriate positioning and fit:

- The splint should cover two-thirds of the length of the forearm.
- The splint should clear bony prominences, and it should not impinge the thumb web space.
- The proximal edge of the splint can be flared for increased comfort.

Modification

Rolyan Deluxe Preformed Resting Pan Mitt Splint (Product Nos. A764-1-6 and A764-19-24—see below for A764-7-12)

Rolyan Preformed Resting Pan Mitt Splint

To trim or modify these splints, which are made from 3/8" (3.2mm) Polyform® splinting material, spot-heat the splint with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for one minute or until pliable. Heating the entire splint may cause it to lose its preformed shape. Polyform has a high degree of drape—for better results when modifying the material, shape it with stroking movements.

Rolyan Mitt Splint with Slot & Loop Strapping

This splint can be modified as described above, but be careful not to heat the splint near the strapping or the plastic base for the slot-and-loop strapping.

Rolyan Deluxe Preformed Resting Pan Mitt Splint (Product Nos. A764-7-12)

Preformed from 1/4" (1.6mm) Polyform Light splinting material, the light splint can be trimmed without heating. If it requires reforming, the area needing modification should be spot-heated with a heat gun or in a heat pan of water at a temperature of

150° to 160°F (65°–70°C) for 30 seconds, or just until pliable. A splint made from Light splinting material will lose its shape rapidly if submerged in hot water.

Rolyan Aquaplast Resting Pan Mitt Splint

Rolyan AirThru Resting Pan Mitt Splint

To trim or modify these splints, which are made from 3/8" (3.2mm) Aquaplast splinting material, spot-heat the splint with a heat gun or in a heat pan of water at a temperature of 160° to 170°F (70°–75°C) for 2 minutes or until pliable. Smooth areas that have been trimmed with the palm of the hand.

Strapping

Apply straps as desired. Only some splints include strapping materials. Suggested strapping for most of the splints is shown in A. An additional strap may be placed across the wrist for increased support.

Rolyan Mitt Splint with Slot & Loop Strapping

—Put the splint into position by sliding the fingers under the elastic strapping. If necessary, adjust the slot-and-loop strapping for proper fit—feed it through the slotted plastic base on the underside of the splint to increase or decrease the amount of strapping needed to secure each finger.

—Attach the two hook tabs to the ulnar and radial borders of the hand portion of the splint. Attach the elastic strap to the hook tabs.

—Attach forearm and thumb straps as shown in B. If desired, trim strap length. An additional strap may be placed across the wrist for increased support.

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Patterson Medical Ltd.