

## Sammons Preston® Quick Clean Gait Belt

Economy, Black, 60" 9268-39

|                    |                       |
|--------------------|-----------------------|
| Black, 60" 5525-45 | Black, 72" 5525-45-01 |
| Blue, 60" 5525-44  | Blue, 72" 5525-44-01  |
| Pink, 60" 5531-98  | Pink, 72" 5531-98-01  |

### Sammons Preston® Quick Clean Gait Belt

Designed to help caregivers while doing transfers and when assisting patients with walking. Being able to get a secure hold point of patients enhances patient safety. The security of gait belts reduces the risk of back injuries for the caregiver. The Sammons Preston Quick Clean Gait Belt is easy to access because caregivers can wear them. To deter cross contamination, it can be quickly disinfected in between uses.

### Contraindications

Not advised for patients who have the following: hernias, G-tubes, colostomy, ostomy and Cardio Obstructive Pulmonary Disease. Other contraindications include (but are not limited to): patients who have surgery incisions or are fitted with monitoring equipment, tubes, cords that could be ineffective because of pressure or pinching against the gait belt. If the gait belt could be unsafe or unsuitable, DO NOT use it.

### Before Use

Inspect the belt closely. If imperfections are found, contact us immediately. **Please read instructions, and if they are not clear, consult a medical professional.** Following your facility's guidelines for patient monitoring is essential to effectiveness and safety.

### Application Instructions

Position the belt all the way around the patient, and tighten by buckling the strap. As you adjust the belt keep in mind that it should remain secure, yet comfortable for the patient.

When done adjusting, you should be able to slip your open hand between the belt and patient. Check the tightness of the belt and buckle so that it will not be able to slip on the patient during use.

### Care

When the belt is not being used, wipe it clean with a mild detergent—no bleach, no heavy-duty cleaning products—to clean. Rinse thoroughly, keeping in mind residual chemicals must be removed. Ensure that the belt is completely dry before next use.

### Warnings

NEVER use this gait belt as a restraint or in a moving automobile.

If either is found, dispose of the belt as a biohazardous item following the policies of your facility.

Test the belt frequently to make sure it is applied securely and will not become loose during the transfer of the patient.

### Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



#### Patterson Medical

1000 Remington Blvd., Suite 210  
Bolingbrook, IL 60440  
Phone: 800-323-5547  
Direct Phone: 630-378-6000  
Fax: 800-547-4333  
Direct Fax: 630-378-6310



#### Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK

United Kingdom  
Phone: 08444 124 330  
Fax: 08448 730 100  
International  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784



© August 2011  
Patterson Medical Products, Inc.  
552545LIT