
Rolyan® Walker Splint:

A767-1 through 4

A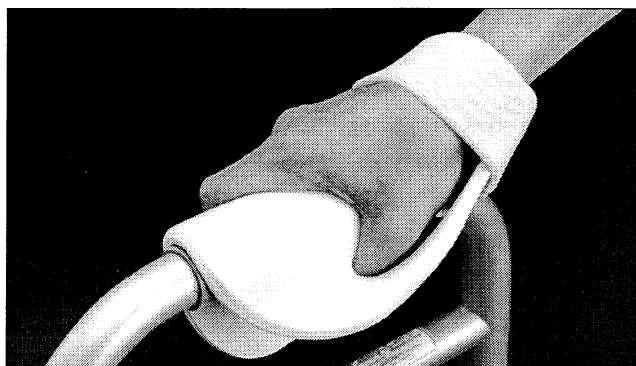

B

INDICATIONS

- To help people with a weak grip hold onto a walker handle

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the strap is not so tight that it interferes with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

- For safety in case the patient falls while using the walker, the splint should not be permanently attached to the walker.

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. Avoid wetting the adhesive on the hook tabs and the optional Kushionflex® pad.
- The strap can be washed with soap and water.
- The splint and strap should be allowed to dry thoroughly before reuse.

PROCEDURE

The splint should be positioned over the handle of the walker so the thumb fits into the side opening (A).

- Attach the hook tabs to the underside of the splint where it curves away from the walker and attach the strap to one of them (B).
- Place the splint over the walker handle. The splint should be snug enough so it does not rotate on the walker handle, but it should be loose enough to be removed without excessive effort.
- If the splint requires adjusting, spot heat the Polyform® splint with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) until pliable. **Heating the entire splint may cause it to lose its preformed shape.**
- The Kushionflex pad can be placed where the heel of the hand rests on the splint. Trim it to fit as necessary before applying the self-adhesive back to the splint.
- A completed splint is shown in A.

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