## Rolyan® D-Ring Wrist Brace:

A607-11 through 30, A608-1 through 10, A609-1 through 10, A611-100 through 109, A611-LXS through RXL, and A915-LXS through RXL

## Rolyan® Workhard® D-Ring Wrist Brace:

A383-11, 12, 21, 22, 31, 32, 41, 42, 51, and 52

## Rolyan® D-Ring Wrist and Thumb Spica Splint:

A615-1 through 10

## Rolyan® Workhard® D-Ring Wrist and Thumb Spica Splint:

A615-200 through 209

A

B

### INDICATIONS

The D-ring wrist braces support the wrist; the D-ring wrist and thumb spica splints support the wrist and thumb.

- For support and protection to allow healing after injury
- For helping relieve pain caused by arthritis
- For helping decrease pain and prevent further injury caused by repetitive activity

### CONTRAINDICATIONS

- Not for treatment of severe sprains
- Not for immediate treatment following fractures
- Not for use directly over open wounds and infected areas

### D-ring wrist and thumb spica splints

- Not for use when IP immobilization is required

### PRECAUTIONS

- If unusual swelling, skin discoloration or discomfort occurs, discontinue use. If pain or discomfort persists, consult a healthcare professional.
- Be careful that the straps are not so tight that they interfere with circulation.

### CARE AND CLEANING

#### Rolyan D-Ring Wrist Brace, Rolyan D-Ring Wrist and Thumb Spica Splint

Remove stay and fasten straps. Hand wash fabric in warm water and mild soap. (The Rolyan D-Ring Wrist Brace and Rolyan D-Ring Wrist and Thumb Spica Splint can also be machine washed.) Do not wring. Dry at room temperature. Be sure the splint/brace is completely dry before reapplying it.

#### Rolyan Workhard D-Ring Wrist Brace and Rolyan Workhard D-Ring Wrist and Thumb Spica Splint

Surface clean the outside of the leather splint/brace using saddle soap. Do not immerse in water.

### APPLICATION

- Be sure the hem portion of the pocket is over the end of the stay.
- Do not insert the hand between the layers of stockinette. Apply the wrist brace as shown in **A**; apply the wrist and thumb spica splint as shown in **B**.
- The stay can be adjusted for proper fit.

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**Patterson Medical, Ltd.**