

AIR TRIO™ Shell Low Top Ankle Walker

APPLICATION INSTRUCTIONS

The decision as to when to apply this brace is at the discretion of the physician or health care professional. Please read these instructions completely to help provide proper fit.



3. - 4.



5.

Patent Pending



6.

1. Choose the correct walker size for the patient. The end of the patient's toes should not extend past the toe end of the walker. Place the **Comfort® Max Walker** sock on the patient
2. Remove the anterior (front) plastic panel from the walker. Open the hook and loop closures on the walker liner in preparation for placing the patient's foot in the liner.
3. Position the patient's foot in the walker liner, sliding the patient's heel in proper position as far back into the heel area of the walker as possible.
4. Secure the liner around the patient's foot and lower leg with the hook and loop closures. Attach the toe cover by folding it over the toes and securing the hook to the liner.

Note: The toe cover can be removed if desired.

5. Position the anterior plastic panel over the walker liner with the hook tabs on the panel holding it in place.
6. Beginning with the toe strap, feed the straps through the loops and secure the hook and loop closures. The straps should then be adjusted to ensure a secure and comfortable fit. Caution should be taken to not over tighten the straps.
7. The AIR TRIO™ liner has dual air chambers that are inflated separately.
8. The inflation tubes are located on the sides of the walker and can be re-positioned on the liner if desired.
9. Inflate the air liner chambers by placing the **CLEAR** end of the inflation bulb into the air tubes and pump with one hand. Both air chambers should be inflated to provide even compression.
10. Caution: **DO NOT OVER INFLATE THE AIR CHAMBERS**
11. To release air from the chambers, place the **BLACK** end of the inflation bulb into the air tubes and pump to deflate.
12. The air chambers and inflation tubes can be removed from the liner if desired.

This product is designed for single patient use. Liner can be hand washed in mild soap and must be rinsed thoroughly. **AIR DRY ONLY.** The Air Trio™ should be worn with the **Comfort® Max Walker** sock. Additional pads included if needed.

Reorder Number	Size	Male	Europe	Female	Europe	Unit
0814 0992	Small	4 - 7	35 ^{1/2} - 37 ^{1/2}	5 - 8	35 - 38 ^{1/2}	Each
0814 0993	Medium	7 ^{1/2} - 11	40 - 45	8 ^{1/2} - 12	39 - 44	Each
0814 0994	Large/XL	11 ^{1/2} Plus	45 Plus	12 ^{1/2} Plus	44 Plus	Each

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121
 651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934
 e-mail sales@birdcronin.com • www.birdcronin.com

©2009 Bird & Cronin, Inc. All Rights Reserved 10000992