

Transfer Disc

Item #926907 – 12”

Item #AA8864 – 15”



Instruction Sheet



Please read the instructions before use of this product.

Purpose:

The Transfer Disc makes it possible for the user to move from one seated position to another without assistance. The Transfer Disc facilitates body rotation with minimal amount of physical exertion. If a caregiver is involved, this product gives a better working position.

To use, the user's feet should be placed on the Transfer Disc. The caregiver assists the user to an upright position, if needed. The user can now be transferred to a new seated position. If a caregiver is not needed, the user can transfer independently without aid.

For safety, both sides of the Transfer Disc are covered with a non-slip rubber material.

Maintenance:

Clean the turntable with domestic cleaning items that do not contain abrasive or corrosive ingredients. Clean the outer side of disc only. Do not use solvents or items containing chlorine. Do not submerge the item in water.

Max weight: 330 lbs (150 kg). Use only as directed.



Sammons Preston
A Patterson Medical Company
1000 Remington Blvd., Suite
210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European
Representative
Homecraft Rolyan
A Patterson Medical Company
Nunn Brook Road
Huthwaite
Sutton in Ashfield
Nottinghamshire
NG17 2HU, UK

UK Customer Services
Tel: 08444 124 330
Fax: 08448 730 100

International Customer
Services
Tel: +44 1623 448 706
Fax: +44 1623 448 784