## Rolyan® Progressive® Palm Protector Splint:

A812-313 through A812-318

## Rolyan® Progressive® Palm Protector Splint with Wrist Support:

A812-301 through A812-306

## Replacement Foam Pieces

A812-307, A812-309, A812-311

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to both of the splints.

Figures referred to in the instructions are on page 6.

### INDICATIONS

- For mild to severe finger spasticity or contractures

### CONTRAINDICATIONS

- Not for use in the presence of severe arthritic deformities
- Not for use with severely fluctuating edema
- Not for use over open wounds and infected areas

### PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The stay will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- Remove foam palm piece, strap with finger separators, and stay if using the wrist support. Fasten hook-and-loop closures. Hand wash splint and strap with finger separators in lukewarm water and mild soap.

Rinse thoroughly. Remove excess moisture with a towel and dry at room temperature. Be sure splint is completely dry before reapplying it.

- Foam palm piece can be surface wiped, but it should not be immersed in water.
- Wash and dry hand before splint application, and have splint off during mealtimes to reduce need for washing.

### INSTRUCTIONS FOR USE

Note that the pile surface goes against the skin, with the felt surface facing out.

#### Rolyan Progressive Palm Protector Splint

1. Place splint in hand to check fit. It should extend no more than ½" (1.3 cm) beyond sides of hand or finger tips. If necessary, mark splint for trimming (A).
2. Remove splint and trim with scissors.
3. Place finger separators on outside of splint and pull the three finger separators through slots in splint. Notches in finger separators should point toward wrist (B).
4. Attach foam palm piece to finger strap on volar surface of finger separators.  
If losing foam palm piece is a concern, use hot glue gun to attach it permanently to finger strap after achieving desired thumb positioning.
5. Put splint in position and check as follows:  
—Check thumb hole for clearance. If necessary, cut hole larger.  
—Check finger separators for proper fit. If they touch web spaces, trim them. If they are too far from finger web spaces, remove finger separators and cut slots in splint slightly longer.
6. Place splint onto hand and fasten wrist strap.
7. Finger strap can be wrapped to incorporate thumb and fingers, only fingers, or only thumb, as follows:  
Thumb and fingers: Starting at thumb, wrap finger strap around volar surface of splint and over dorsal surface of fingers. Attach hook piece near thumb (C).

Only fingers: Wrap finger strap from index finger over dorsal surface of fingers and attach it to hook piece on strap at ulnar side of hand (D). Trim excess length of finger strap.

Only thumb: Pull finger strap over thumb and around volar surface of splint and attach it to hook piece on strap at ulnar side of splint (E). Trim excess length of finger strap.

8. As degree of contractures or spasticity decreases, foam palm piece can be replaced with larger foam palm piece. Simply remove foam palm piece from finger strap and secure new one in its place.

#### Rolyan Progressive Palm Protector Splint with Wrist Support

9. Prepare splint for applying to hand by following instructions 1 and 2.
10. Check fit of forearm cuff. In addition to clearing elbow fold and ulnar styloid process, splint will be more comfortable if ends of forearm cuff overlap no more than 1½" to 2" (3.8 to 5.1 cm).
11. Without allowing them to touch each other, place the two **Ezeform®** stay pieces in a heat pan of water heated to 160° to 170°F (70–75°C) for 1 to 2 minutes or until pliable. Remove from water and pat dry with a towel.
12. Line stay pieces up carefully and place one piece on top of the other so they bond together to form a single stay.
13. With wrist in appropriate degree of extension, mold stay as shown in F. For greater comfort, flare both ends of stay.
14. Put stay in pocket and continue fitting splint as described in instructions 3 through 5.
15. Place splint on hand and fasten D-ring straps.
16. To complete splint application, follow instructions 7 and 8.

(Original design courtesy of Lynn Swedberg, MS, OTR, and Barb Biles, COTA.)

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**Patterson Medical Ltd.**